

200 Kg Shower Stool (With Telescopic Legs)



Shower stool made for large individuals and users that require additional space.

The 70 cm seat width makes it big enough to accommodate all sizes of persons comfortably.

It has been specially designed to withstand a max. load of 200 kg.

The height of the stainless steel telescoping legs is adjustable with three seat heights: 47, 50 and 53 cm.


You can easily adjust the height using cross clips without any tools at all.

The seat can be inclined at an angle using the telescoping legs (at 7° and $3,5^{\circ}$ angles), making it easier to sit down and get up.

You may want to use a seat height of 47 cm at the front and 53 cm at the back.



HMN since 1934

Technical Data: 200 Kg Shower Stool (With Telescoping Legs)	
Seat height	47 cm - 50 cm - 53 cm
Max. total height	67 cm
Min. total height	61 cm
Seat width	70 cm
Max. width at floor	83 cm
Min. width at floor	82 cm
Seat depth	36 cm
Max. depth at floor	48 cm
Min. depth at floor	46 cm
Weight	9,4 kg
Max. Load	200 kg
Inclining the seat	With the front clip placed in the lower hole and the rear clip in the upper hole, the seat height is 47 cm at the front and 52 cm at the back (7° angle) With the front clip placed in the lower hole and the rear clip in the intermediate hole, the seat height is 47 cm at the front and 50 cm at the back (3,5° angle)
Surface treatment	Rilsan which is a nylon coating with following characteristics: Waterproof. Chemically proof Proof against salt mist for over 2000 hours Physiological harmless
Slates	Plastic slates 60 mm width and 6 mm thick
Rubber Feet's	Non-slip technical grey rubber "no marks at the floor". Hole in the rubber feet for outflow of water
Telescopic legs	Stainless steel with 3 adjustments, distance 3 cm
Description:	Art. no
200 Kg shower stool 500 mm	310145
Back rest for 200 kg shower stool	310144
Clip for this model shower stool	310065
Backpart	310144
 <p>This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.</p>	
07.10.15	



HMN since 1934