Carla

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
Bruksanvisning Carla

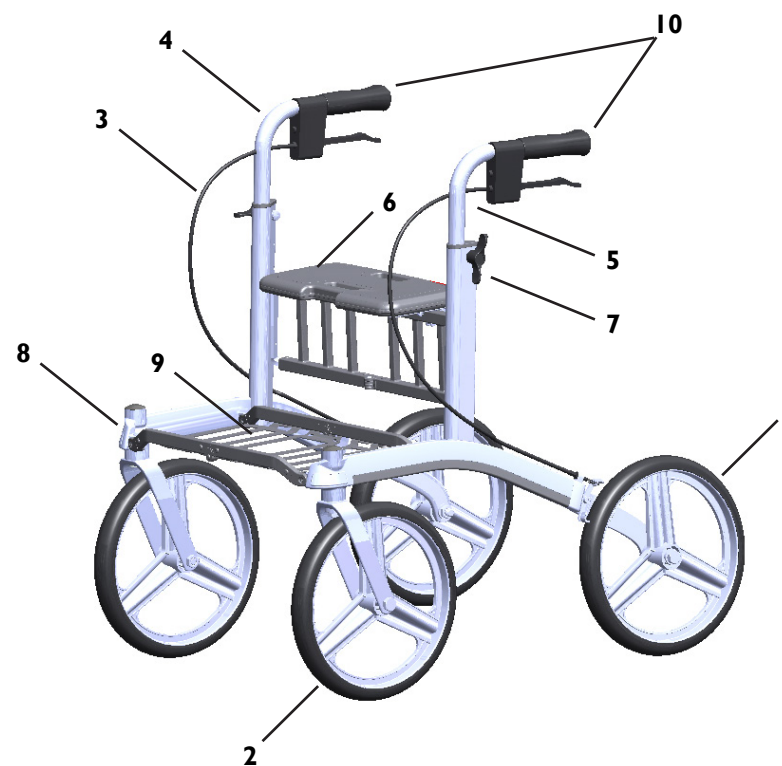Carla

To secure that the walking-aid will function safely for the user, once per year review of the walking-aid should be done.

Cleaning

Painted areas, wheels and handles should be cleaned with a regular cleansing solution, without solvent.

Note! Do NOT use thinner, petrol or other solvents.

Post treatment

After washing, all connections, tight spaces and drainage holes should be blown dry and if needed sprayed with thin oil.

Wing nut

Control.

CE-marking

Make sure that the CE-mark is still there and readable.

Wheel

Make sure the wheels are not worn-out and are spinning easily (without any noise).

Reflectors

Make sure the reflectors are still there and visible.

The Paint

Control the paint for possible damages.

Stability

Control that all wheels are steadily on the ground and that there are no chassis distortion.

Chassi

Control weldings and screw attaches. By larger corrosion the rollator shall be changed.


Lubrication

Gatebushings and hinges should be working without any noise. Lubricate when needed.

Front forks

Make sure that the ball bearing is firmly attached and that the forks swings easily.

Note! The nut on the fork must not be touched.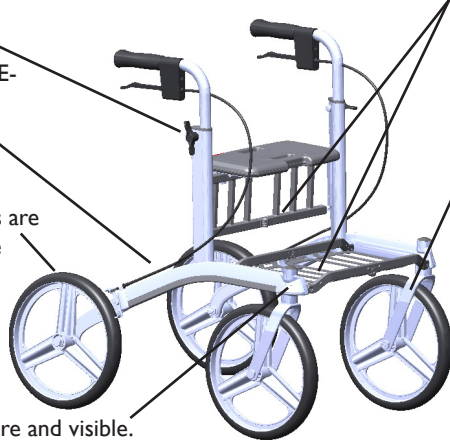

Handle

Must be fixed. Make sure the handle pipes can be pushed down to the chassis.

Brakes

Control that both brakes and parking brakes are working. Wires shall be intact.

Seat


Make sure the seat is kept in place and the screws are tightened. For more details, see back page.

Folding mechanism

Should be intact and the locking device should function.

Test driving

Make a final control by test driving.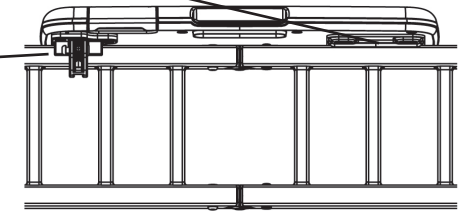

For more details, see back page.

Note!

Recondition must be done by a person how has good knowledge about how the job should be performed. An incorrect performed recondition could result an accident/ incident. Don´t hesitate to contact your retailer if questions come up.

When exchanging parts of the rollator, use only Human Care parts to ensure continued CE-validation.

When changing the wheel locking liquid shall be applied on the nut and screw.

If the corrosion is more than a film of rust the rollator shall be discarded and return for recycling.

Intended use

The rollator should only be used as support whilst walking or as seating whilst stationary. Do not alter the rollator yourself.


Product number:	
Owner:	
Notes:	
Attended by:	Date:

Spareparts

No	Designation	Quantity	Art. no.	Other information
1	Back wheel complete Wheel with bearing and distance tube Locking nut Pin bolt	1	70850	
2	Front wheel complete Wheel with bearing and distance tube Locking nut Pin bolt	1	70849	
3	Wire complete Housing/Wire	1	70239	
4	Right handle complete Handle pipe/handle ERGO/brake Handle pipe/handle ERGO/brake	1 1	70506 70182	For 63 cm For 58 cm
5	Left handle complete Handle pipe/handle ERGO/brake Handle pipe/handle ERGO/brake	1 1	70505 70082	For 63 cm For 58 cm
6	Seat complete Seat pad/Gate lock	1	70090	
7	Wing nut Wing nut Screw Washer	1 1 1	20-052 70208 000124	
8	Reflex Reflex right Reflex left	1 1	70550 70650	
9	Gate complete Gate Gate	1 1	70852 70851	For 63 cm For 58 cm
10	Handle Handle ERGO	2	20-024	

Other

User manual Carla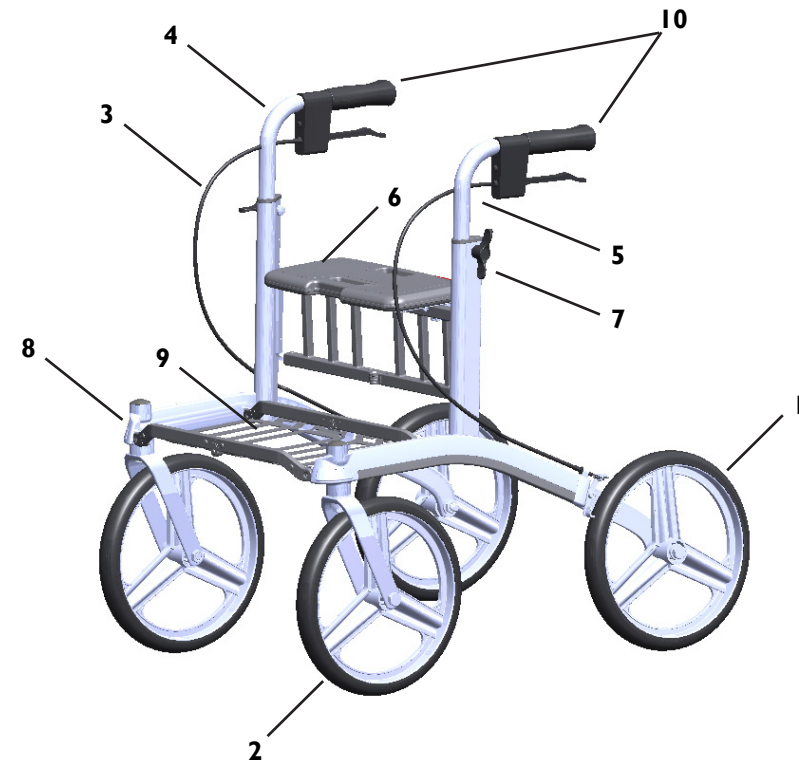

• Fabriksvägen 2, SE-245 34 Staffanstorps, Sweden •
 • Tel. +46(0)8 - 665 35 00 • Fax +46(0)8 - 665 35 10 •
 • www.humancare.se •