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EN: IMPORTANT!

- WHEN CHARGING, DO NOT TURN ON THE SWITCH BEFORE THE CHARGER IS CONNECTED TO THE CHAIR.
- THE BATTERY SHOULD BE CHARGED 24 HOURS THE FIRST TIME.
- WHEN THE BATTERY HAS RUN DOWN THERE IS A BEEP SOUND FROM THE CONTROL BOX WHEN THE ELECTRICAL PARTS ARE IN USE.
- WHEN THE BEEP SOUND COMES, YOU HAVE TO RECHARGE THE BATTERY AS IT CAN BE DAMAGED IF YOU LET IT RUN 100 % OUT OF POWER.
- IF THE BATTERY HAS RUN OUT OF POWER, YOU CANNOT USE THE ENGINE BY PUTTING THE CHARGER IN THE WALL PLUG AND THE CHAIR. A FULL RECHARGE TAKES TYPICALLY 8 HOURS.
- IF THE LIGHTBOLT ON THE CHARGER LIGHTS GREEN WHEN CONNECTED, IT IS DUE TO A MALFUNCTION OF THE BATTERY, AND THEREFORE IT NEEDS TO BE CHANGED.
- DO NOT CHARGE THE CHAIR WHILE BEING USED IN AREAS WITH WATER.
- DO NOT USE THE ELECTRICALL FUNCTIONS ON THE CHAIR WHILE SPRAYING WITH WATER.

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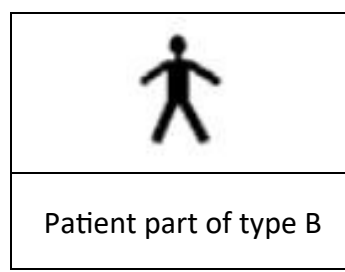
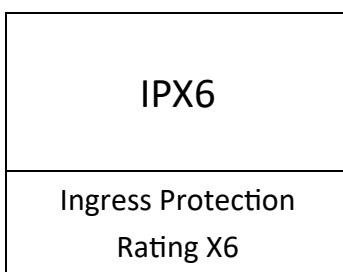
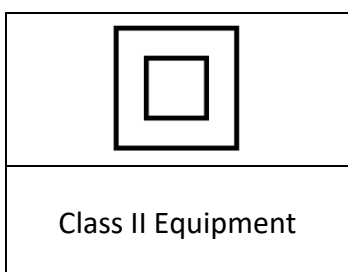
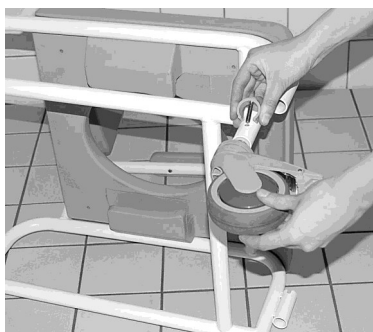
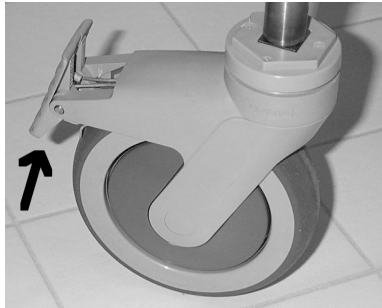
EN: Adjusting the back (see photos). Important! Back wash max. at 80 degrees.

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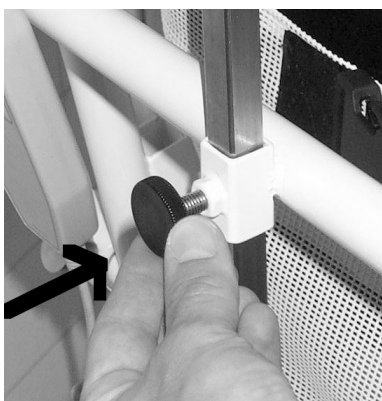
EN: The headrest is adjustable in height by loosening and tightening the screw at the right position.

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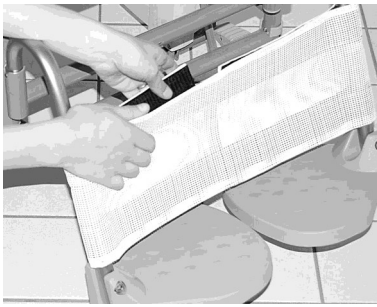
EN: The calf support is secured and adjusted by the Velcro strip on the back. Important! Calf support washed at max. 80 degrees.

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EN: When mounting the armrest the snap lock is lifted, then the armrest pulled up and pushed to the side.

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EN: Cleaning

The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.

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EN: Complaint rights for frame

HMN grants 5 years complaint rights against frame break-down for all M2 Shower/commode chairs and stainless steel shower stools. The complaint rights applies for production and material break downs, and can only be used if the products have been in normal use. Please note that the normal wear and tear is not covered under complaint rights.

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Preventative Maintenance Check:

Applies to the following Art.no: (310216, 310217)

To ensure that the aid functions safely for the user, we recommend that the following inspections be carried out.

1. Check that the frame is without breaks and the chair is intact.
2. Check that the wheels on the chair can roll freely.
3. If the chair has a directional wheel, check that it functions normally.
4. Check that the brakes on the wheels work.
5. Check that the clips for height adjustment of the chair's wheels are not defective.
6. Check that the seat is intact and there are no breaks in the PU foam.
7. Check that the armrests are correctly mounted, function, and there are no breaks in the PU foam.
8. Check that the backrest in netweave is correctly mounted and intact.
9. Check that the chair's tilt function works.
10. Check that the chair's foldable back works.
11. Check that the charger works correctly (see possibly the user manual).
12. Check that the battery can be charged (see possibly the user manual).

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EN:

This product should be recycled following national regulation. After disassembly of the product, sort the components based on what they are made of and recycle them individually. Please contact your local recycling place for guidance if necessary.

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