- DK:** Brugsanvisning for **bade-/toiletstol M2 Multi-Tip & Multi-Tip Big med joystick.**
Beregnet til brug ved badning/brusning og toiletbesøg
- EN:** User guide for **shower/commode chair M2 Multi-Tip & Multi-Tip Big with joystick.**
Intended for use at bathing/showering and toilet visit
- DE:** Gebrauchsanweisung für **dusch-/toilettenstuhl M2 Multi-Tip & Multi-Tip Big mit Joystick.**
Stuhl für Dusch/Toiletten Gebrauch
- SV:** Bruksanvisning för **dusch-/toalettstol M2 Multi-Tip & Multi-Tip Big med joystick.**
Avsedd att användas vid bad/ dusch och toalettbesök
- NO:** Brukarveiledning for **dusj-/toalettstol M2 Multi-Tip & Multi-Tip Big med joystick.**
Designet for bruk når badning/ dusjing og toalettbesøk
- FI:** Käyttöohje että **kylpy/suihku tuoli M2 Multi-Tip & Multi-Tip Big joystickilla.**
Suunniteltu käytettäväksi uiminen/suihkussa ja WC: ssä
- NL:** Aanwijzingen voor **douche/toiletstoel M2 Multi-Tip & Multi-Tip Big met joystick.**
Ontworpen voor gebruik bij het baden/douchen en toiletbezoek

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A	Side 6	Generel information / Mål
B	Side 6	Montering
C	Side 7 -15	Håndbetjeningen og batteri
D	Side 16	Brug af stolen
E	Side 17	Generelt
F	Side 17-18	Sikkerhed
G	Side 19-22	Elektrisk system

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EN: IMPORTANT!

When receiving a new chair do the following:

1. Fully charge the batteries
2. Reset the system (see users guide for how to reset)

(Lithium batteries are sensitive regarding cold surroundings so if you keep the chair in a cold place then please follow the above instruction as well before use)

The power plug is under the white cover on the front of the chair. Insert the plug in the power socket for charging. The white cover must always be mounted when bathing. The cover plate is taken on and off without the use of tools.

If the height adjustable engines does not run simultaneous, it can damage the chair. Activate the emergency lowering (see page 17) and lower the chair to the lowest point before releasing the button. NB! Do not use the electrical functions on the chair while spraying with water!

NB! Please note that if the power plug is wet when bathing and cleaning, wipe it completely before charging! If the power plug is wet, there is a possibility that the relay will switch off.

NB! Do not use in sauna! M2 Multi-Tip joystick standard does not fit over a toilet.

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



	<p>Class II Equipment</p>
<p>IPX6</p>	<p>Ingress Protection Rating X6</p>
	<p>Patient part of type B</p>

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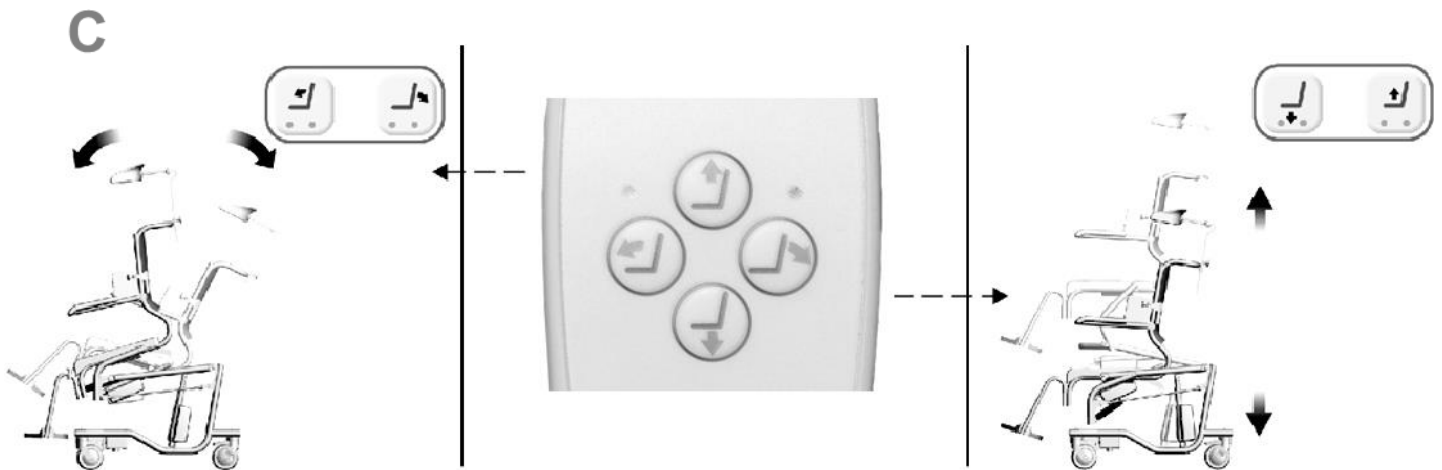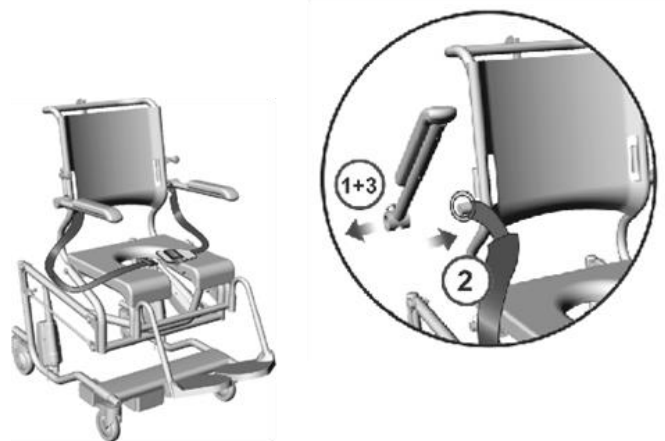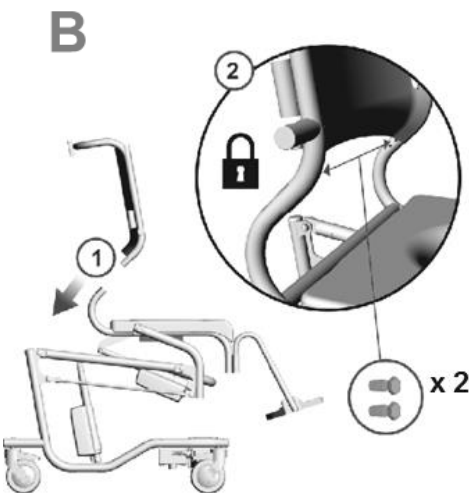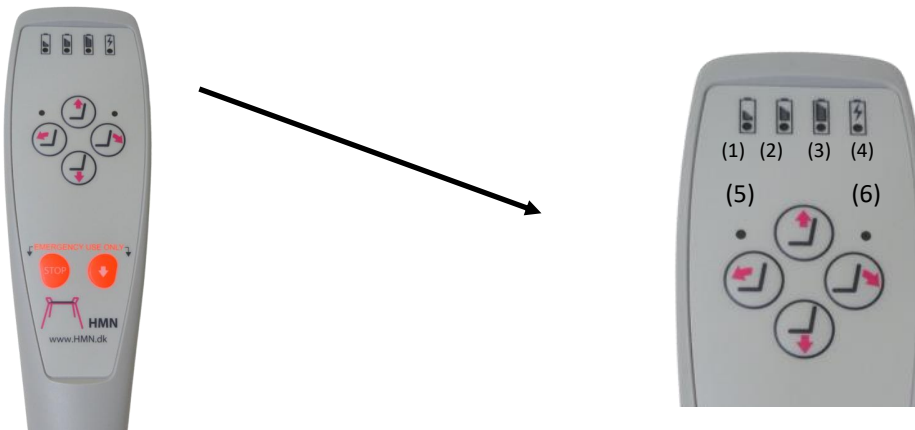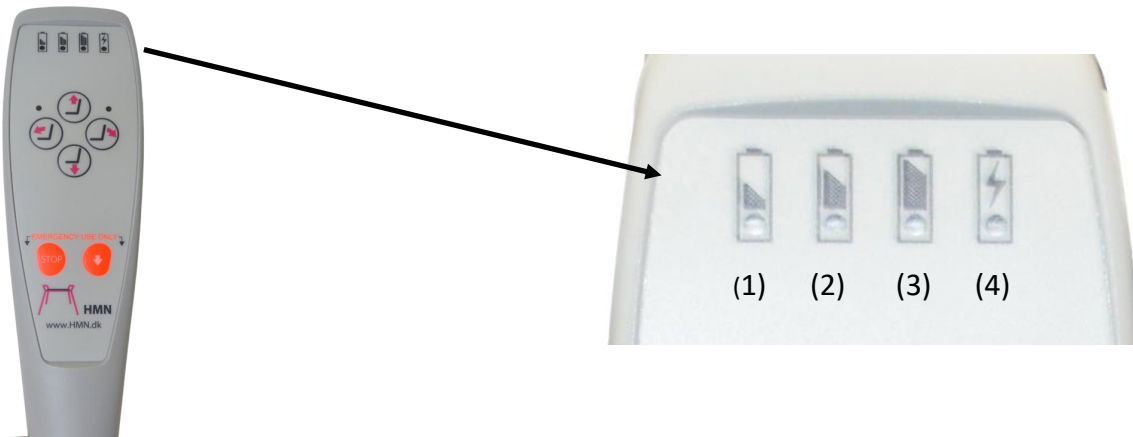
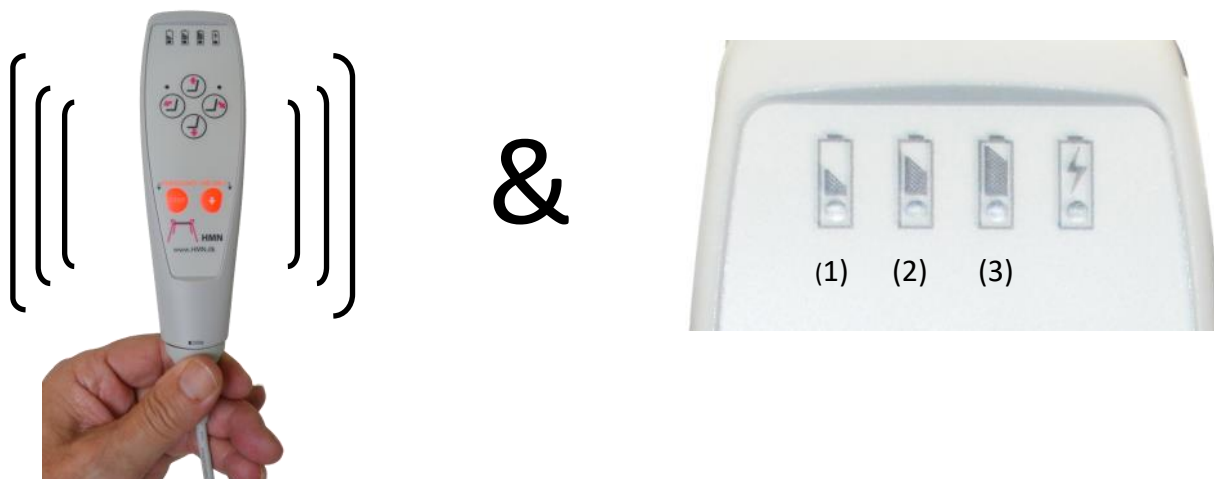
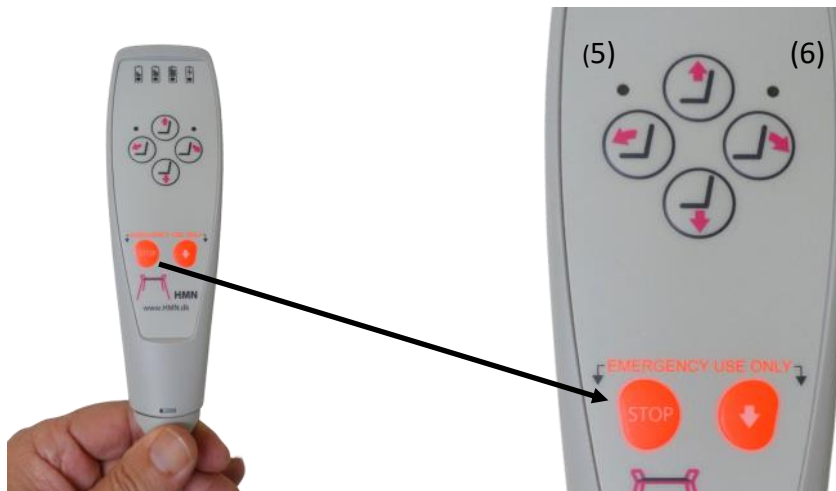
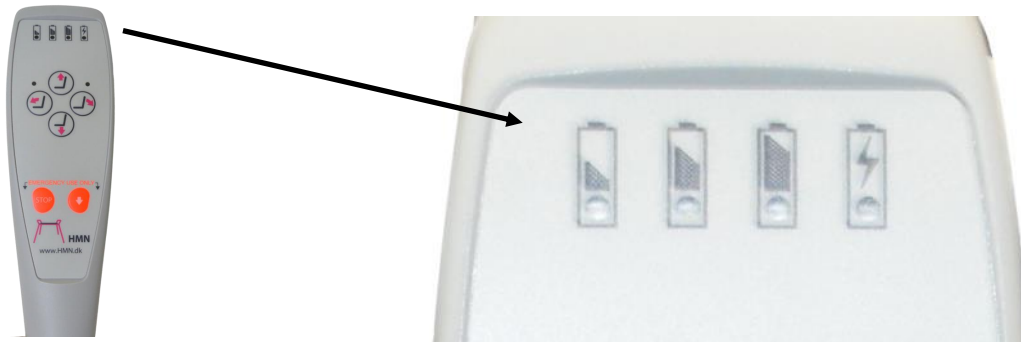
EN: About battery. Upon receipt of a new chair: The chair is fully charged in connection with production, but since there may be a power loss during storage / transport, the chair should not be expected to work before a full charge (= 12 hours or until the battery light stops lighting).

Regarding power loss. If the chair is not in use: At temperatures from about +10 degrees Celsius to +30 degrees Celsius: Expect a power loss per day of approx. 1½ to 2½ %. At temperatures below 10 degrees Celsius: Expect that the battery can be discharged in less than a day.

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EN: Using the joystick.

The chair is easily controlled by the influence of the joystick in the direction you want the chair.

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EN: Location of the joystick. The joystick can be placed both in front of the user or behind the user. Using the thumbscrew, attach the joystick where desired.

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EN: You set the driving speed of the chair by either turning left to decrease speed or right to increase speed.

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EN: Avoid sitting on the last 10 cm of the seat, as the chair may tip forward.

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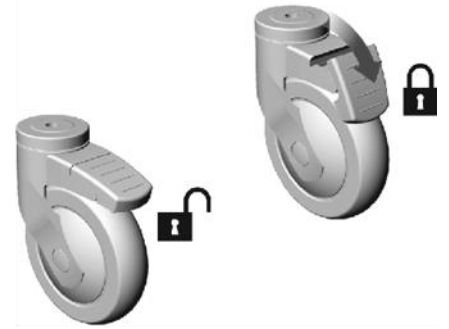
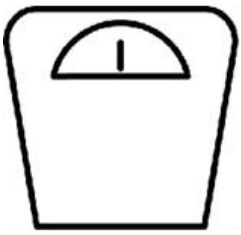
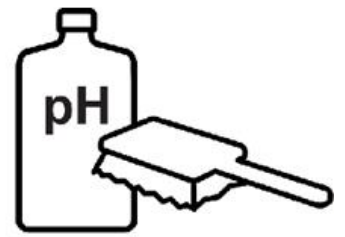
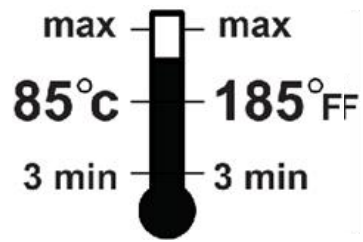
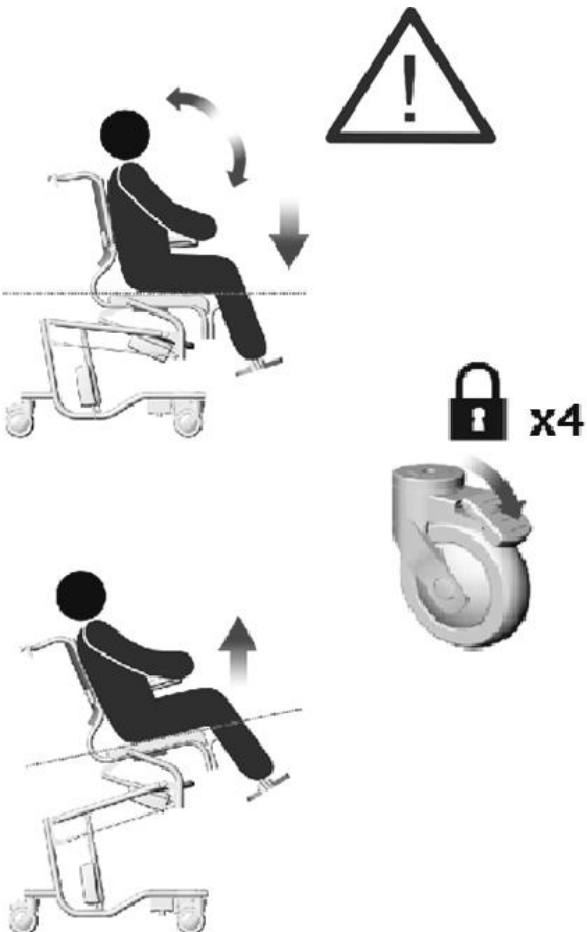
EN: If the user cannot hold on to the armrests, the hands must be placed in the lap while operating the chair.

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
Trouble Shooting/guide for electrical issues with M2 MULTI-TIP chairs.

NB! AS A FIRST COURSE OF ACTION CONCERNING ELECTRICAL ISSUES, ALWAYS TRY TO CHARGE THE CHAIR FOR 24 HOURS!

<p>1. If only the left/last battery light is on.</p>	<p>The chair is low on power and must be charged. The chair will not elevate if only 1 battery light is on.</p>
<p>2. If the 2 smaller lights at the arrow buttons on the hand control is on.</p>	<p>The emergency STOP has been activated. It must be deactivated by holding down the emergency stop button on the hand control until the chair beeps once and the 2 small diodes at the arrow keys go out.</p>
	<p>3. All 4 lights at the top of the hand control is blinking.</p> <p>The chair's control box has received an error signal. The chair must be reset. This is done by first pressing the emergency stop button on the hand control until the emergency stop is activated, and then holding the button next to the emergency stop button. First the chair will beep 3-4 times, then it will lower the chair completely. When the chair is stationary, it is reset.</p> <p>If the chair after this is still blinking in all 4 top lights on the hand control it means there is a defect component.</p> <p>Contact HMN for further advise.</p>

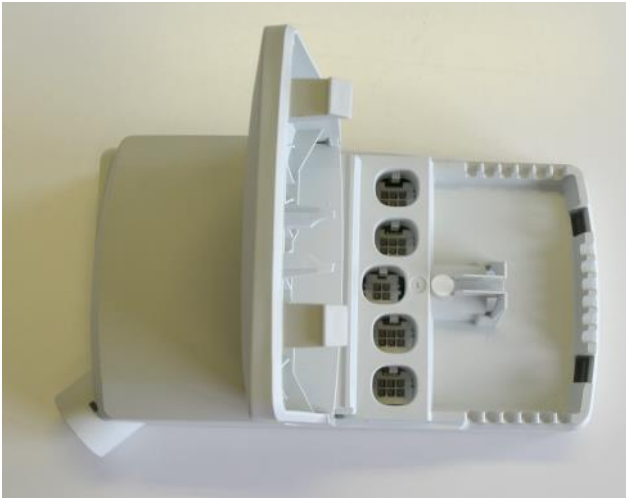4. If none of the lights on the hand control is on (after pressing any of the buttons), and the chair can't elevate or tilt.

Check that all the wires are correctly plugged in. Try charging the chair for 24 hours after checking the wires.

If the problem continues after this it might be a defect battery or other electrical components.

Contact HMN for further advise and trouble shooting.

The picture on the left shows the control box:

Issues charging the chair.

1. If the chair wont charge after hours of trying.

Check that the outlet you are using has power in it, and that it is turned on.

2. The chair won't start charging, and the light indicating charging on the hand control is not blinking.

At the top and to the right on the hand control is a light indicating charging status. This light will blink if the battery is charging. At first when you plug the wire for charging in to an outlet this light will not blink, it will just light up.

After some time the light will start to blink, and this indicates that the chair is charging. The reason some time might pass before it starts charging is that the system is prepping the battery to receive power. This might take extra time if the battery is cold, or the chair is very low on power. Try to reset the system as described above in the first passage. If the problem continues contact HMN for further advice.

3. The charging light on the hand control is not lighting up or blinking even if the chair is plugged in to an outlet.

Check that the charging wire connected to the control box is correctly plugged in. Check that the red locking ring is mounted inside the plug on the control box. This issue is normally because the charging wire or the locking ring is incorrectly mounted. It can also be a faulty battery.

If the red locking ring is mounted correctly it can be seen through the plug like on the picture to the left:

Periodic errors.

<p>1. The chair won't charge every attempt.</p>	<p>If experiencing periodic errors always try and reset the chair as described above.</p> <p>It is also possible that the issue is an incorrectly mounted wire, so check the wires is plugged correctly in.</p> <p>If the problem continues it might be a defect switch box. Contact HMN for further advice.</p>
<p>2. The chair will only work periodically, even after a reset.</p>	<p>If you often experience issues with the chair not working and the resets only work temporarily before all lights in the hand control are blinking again it might be sign of incorrectly mounted wires or issues with the switc box. Contact HMN for further advice.</p>

The chair is elevating uneven, one defect actuator.

<p>1. If one of the actuators on the chair isn't elevating and the chair is being pulled sideways.</p>	<p>The actuators on the chair will not always go at the same speed. This might result in slightly tilted seat frame. In this case just lower the chair completely to reset the pistons in the actuators.</p> <p>If there is a problem with one actuator not working it might pull the chair sideways, and this indicates there could be a problem with that actuator.</p> <p>Do NOT try to elevate the chair if this is the case.</p> <p>With this issue we need to know if the frame has been pulled out of shape. Send us pictures of the frame from the front and back, and we will be able to tell if the frame is damaged.</p> <p>If not, just changing the actuator is enough, but if there is damage to the frame, we might need the chair send to us for repair, or will send you the needed parts to replace. With these issues always contact HMN.</p>
--	--

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EN: Art. nr.

Multi-Tip joystick Horseshoe seat: 310808

Multi-Tip joystick Standard seat, closed front: 310807

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Max. Load 150 kg

(Multi-Tip Big) Max. Load 200 kg

Attention! Push the Clips to the bottom when
adjusting heights.

2 Years complaint rights against
manufacturing errors. 1/2 year on batteries.

Terms of claim for fraame.

HMN grants 5 years claim rights against frame
break-down for all M2 Shower/commode chairs
and stainless steel shower stools. The warranty
applies for production and material break downs,
and can only be used if the products have been in
normal use. Please note that the normal wear and
tear is not covered under warranty.

Cleaning:

The product can be cleaned with regular cleaning
without non-abrasive detergent. Wash and dry the
product after the cleaning. The product can be
washed up to 85° degrees in 3 minutes.

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