
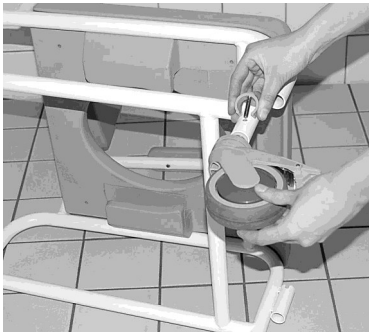
- DK:** Brugsanvisning for **bade-/toiletstol M2 Mini El-Tip og Gas-Tip**
Beregnet til brug ved badning/brusning og toiletbesøg
- EN:** User guide for **shower/commode chair M2 Mini El-Tip and Gas-Tip**
Intended for use at bathing/showering and toilet visit
- DE:** Gebrauchsanweisung für **dusch-/toilettenstuhl M2 Mini El-Tip und Gas-Tip**
Stuhl für Dusch/Toiletten Gebrauch
- SV:** Bruksanvisning för **dusch-/toalettstol M2 Mini El-Tip och Gas-Tip**
Avsedd att användas vid bad/ dusch och toalettbesök
- NO:** Brukarveiledning for **dusj-/toalettstol M2 Mini El-Tipp og Gas-Tipp**
Designet for bruk når badning/ dusjing og toalettbesøk
- FI:** **Käyttöohje suihku- ja wc-tuoli M2 GAS-TIP**
Tuolit on tarkoitettu kylpy-, suihku- ja wc-käyttöön.
- NL:** Aanwijzingen voor **douche/toiletstoel M2 Mini El-Tip en Gas-Tip**
Ontworpen voor gebruik bij het baden/douchen en toiletbezoek
- IT:** Manuale d'uso per sedia da **doccia/comoda M2 Mini El-Tip e Gas-Tip**
Destinato all'uso durante il bagno/la doccia e la visita alla toilette

- DK:** Ved montering af hjul føres teleskopbenene op i rørene og clipsene monteres i passende og ensartet højde.
- EN:** When mounting the wheels, the telescopic legs is placed in the pipes and the clips is mounted in suitable and uniform height.
- DE:** Wenn der Montage des Rades sein Teleskopbeine in den Rohren und die in geeigneten und gleichmäßige Höhe montiert Clips.

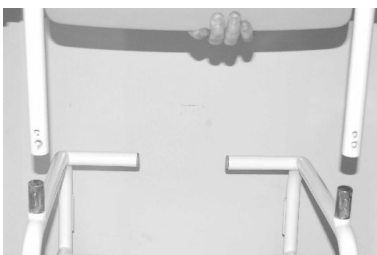- SV:** Vid montering av hjulet, vara teleskop benen upp i rören och klippen monterade i lämplig och lik formig höjd.
- NO:** Når du installerer rattet, være teleskopiske ben opp i rør og klippene monteret i passende og jevn høyde.
- FI:** Asennettaessa rengasta, on teleskooppijalat ylös putkien ja leikkeet asennetaan sopivaan ja yhtenäisen korkeus.
- NL:** Bij het installeren van het wiel, zijn uitschuifbare benen omhoog in de leidingen en de clips gemonteerd in geschikte en uniforme hoogte.
- IT:** Durante il montaggio delle ruote, le gambe telescopiche vengono posizionate nei tubi e le clip vengono montate ad un'altezza adeguata e uniforme..

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- EN:** Screws are tightened using the Allen wrench when mounting the back. Make sure the screws are tight to ensure stability.

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EN: The screws are tightened using an Allen wrench. Make sure the screws are tightened securely to ensure that the spine stability is assured. Remember to tighten your back after about 1 month. The screws should be discarded when dismantling, and will use new screws when next on mounting.

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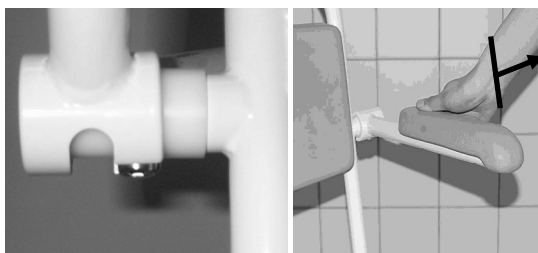
EN: For installing / removing the armrests; tag armrests up and out to the side.

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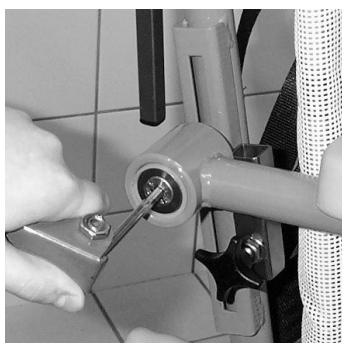
EN: Adjust the height of the armrests by loosening and tightening the Allen screw.

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EN: When tipping the Gas-Tip: tip the chair by grasping the handle. This allows you to activate the gas spring while tipping up or down. If the user is unable to hold his armrests, the helper must put the user's hands in the lap before the chair is tilted.

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EN: Cleaning

The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.

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EN: Complaint rights for frame

HMN grants 5 years complaint rights against frame break-down for all M2 Shower/commode chairs and stainless steel shower stools. The complaint rights applies for production and material break downs, and can only be used if the products have been in normal use. Please note that the normal wear and tear is not covered under complaint rights.

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
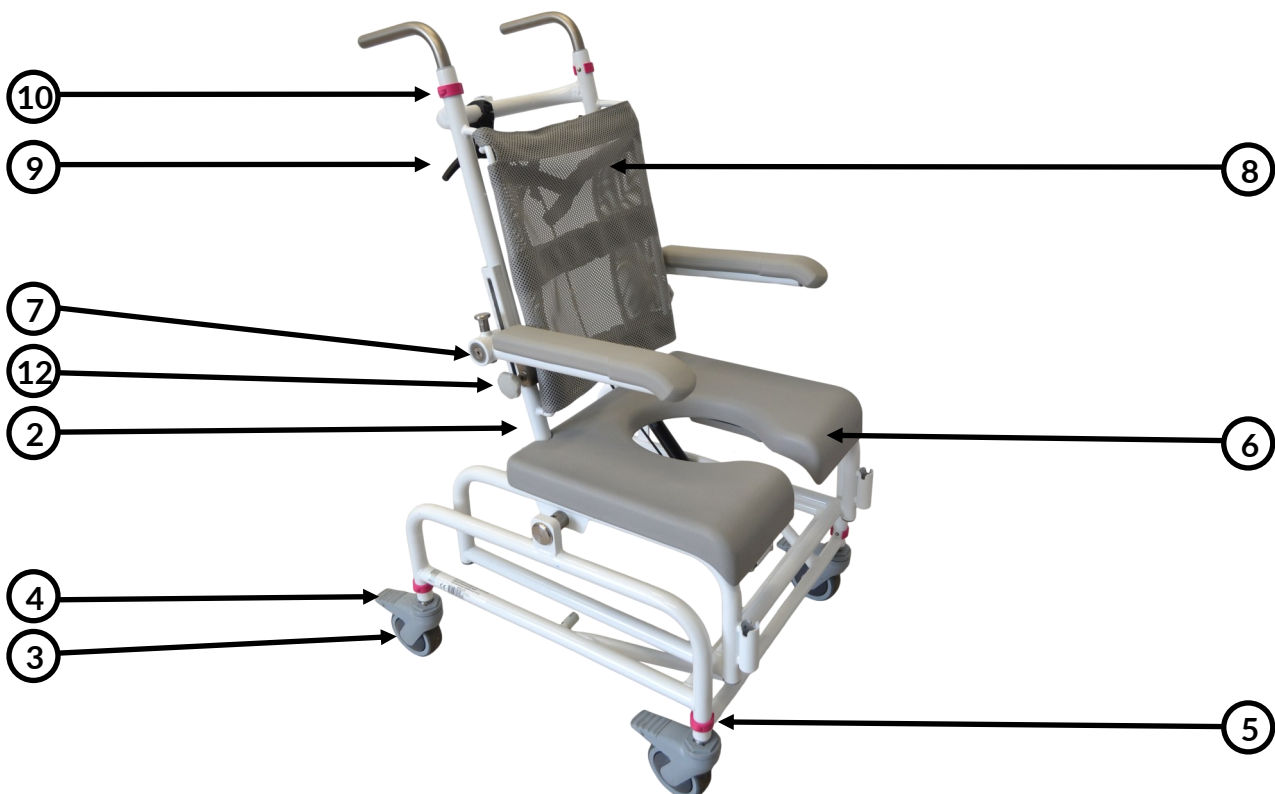
Preventative Maintenance Check:

Applies to the following Art.no: (310503-B)

To ensure that the aid functions safely for the user, we recommend that the following inspections be carried out.

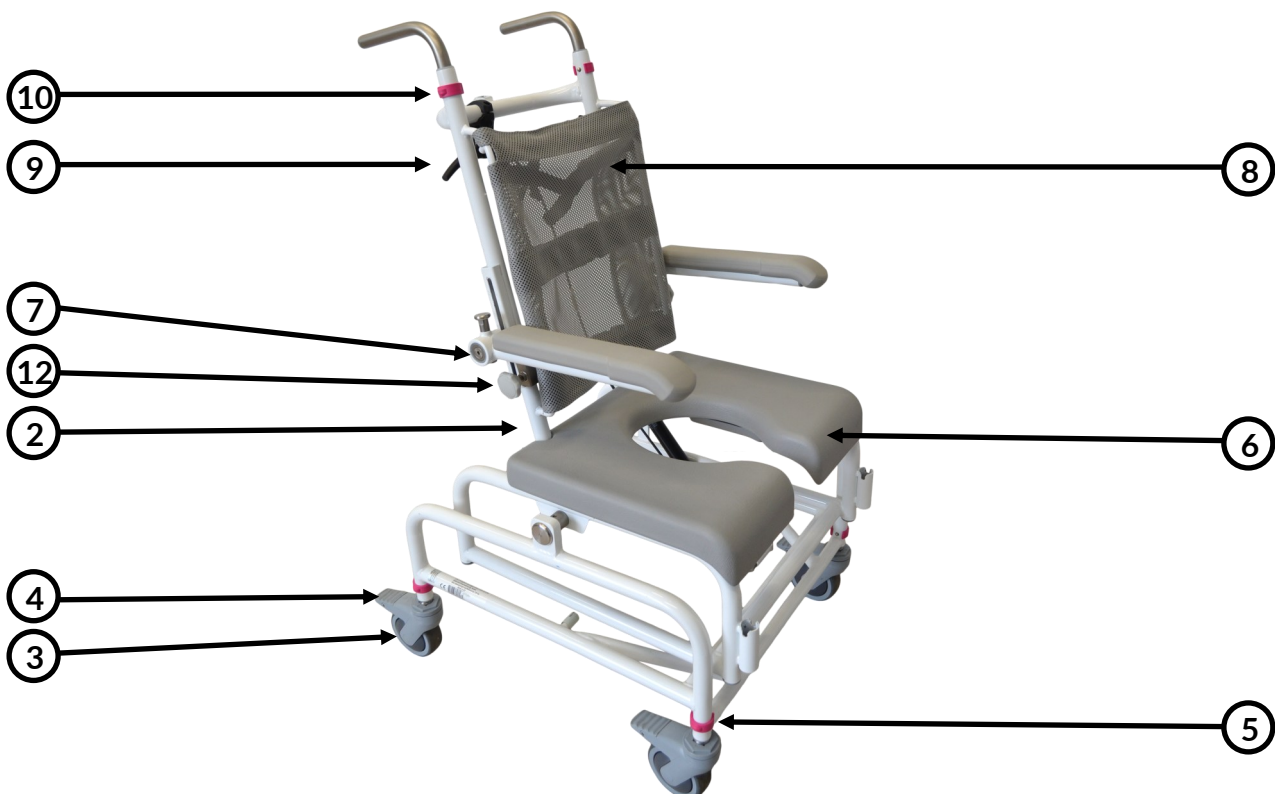1. Check that the frame is without breaks and the chair is intact.
2. Check that the back frame is correctly mounted with 2 screws on each side.
3. Check that the wheels on the chair can roll freely.
4. Check that the brakes on the wheels work.
5. Check that the clips for height adjustment of the chair's wheels are not defective.
6. Check that the seat is intact and there are no breaks in the PU foam.
7. Check that the armrests are correctly mounted, function, and there are no breaks in the PU foam.
8. Check that the back in netweave is correctly mounted and intact.
9. Check that the tilt function works on the chair.
10. Check that the cable for the tilt function is not defective or incorrectly adjusted.
11. Check that the driving handles can be adjusted in height and are correctly mounted with clips.
12. Check that the armrests can be adjusted in height and can be tightened.

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EN:

This product should be recycled following national regulation. After disassembly of the product, sort the components based on what they are made of and recycle them individually. Please contact your local recycling place for guidance if necessary.

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