- DK:** Brugsanvisning for **bade-/toiletstol M2 Multi-Tip Big 200 Kg**
Beregnet til brug ved badning/brusning og toiletbesøg
- EN:** User guide for **shower/commode chair M2 Multi-Tip Big 200 Kg**
Intended for use at bathing/showering and toilet visit
- DE:** Gebrauchsanweisung für **dusch-/toilettenstuhl M2 Multi-Tip Big 200 Kg**
Stuhl für Dusch/Toiletten Gebrauch
- SV:** Bruksanvisning för **dusch-/toalettstol M2 Multi-Tip Big 200 Kg**
Avsedd att användas vid bad/ dusch och toalettbesök
- NO:** Brukarveiledning for **dusj-/toalettstol M2 Multi-Tip Big 200 Kg**
Designet for bruk når badning/ dusjing og toalettbesøk
- FI:** Käyttöohje **suihku-/wc-tuoli M2 Multi-Tip Big 200 Kg**
Tarkoitettu käytettäväksi peseytymisen ja WC-käyntien yhteydessä
- NL:** Aanwijzingen voor **douche/toiletstoel M2 Multi-Tip Big 200 Kg**
Ontworpen voor gebruik bij het baden/douchen en toiletbezoek
- IT:** Manuale d'uso per sedia da **doccia/comoda Multi-Tip Big 200 kg**
Destinato all'uso durante il bagno/la doccia e la visita alla toilette

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EN: IMPORTANT!

When receiving a new chair do the following:

1. Fully charge the batteries
 2. Reset the system (see users guide for how to reset)
- (Lithium batteries are sensitive regarding cold surroundings so if you keep the chair in a cold place then please follow the above instruction as well before use)

The power plug is under the white cover on the front of the chair. Insert the plug in the power socket for charging. The white cover must always be mounted when bathing. The cover plate is taken on and off without the use of tools.

If the height adjustable engines does not run simultaneous, it can damage the chair. Activate the emergency lowering (see page 17) and lower the chair to the lowest point before releasing the button.

NB! Do not use the electrical functions on the chair while spraying with water!

NB! Please note that if the power plug is wet when bathing and cleaning, wipe it completely before charging! If the power plug is wet, there is a possibility that the relay will switch off.

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


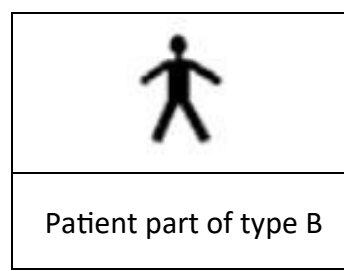
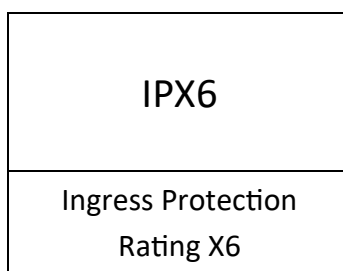
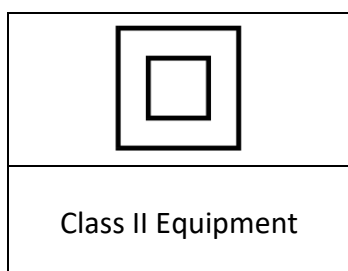
Voltage and current information i.e. 100-240

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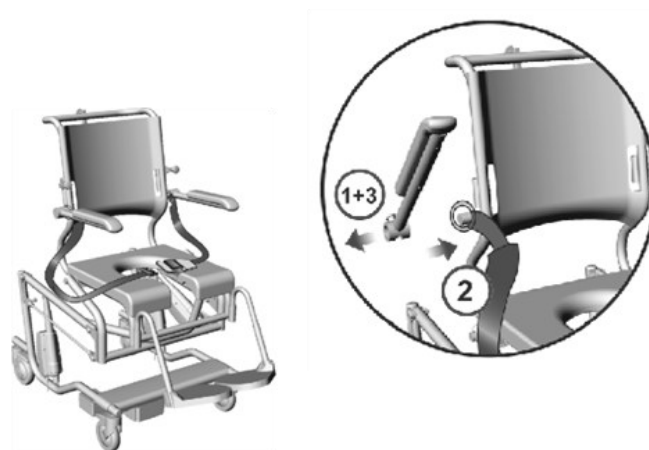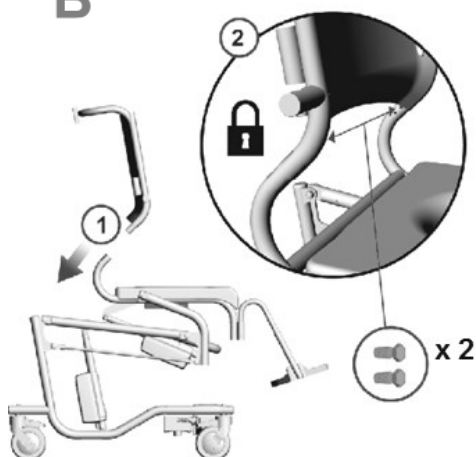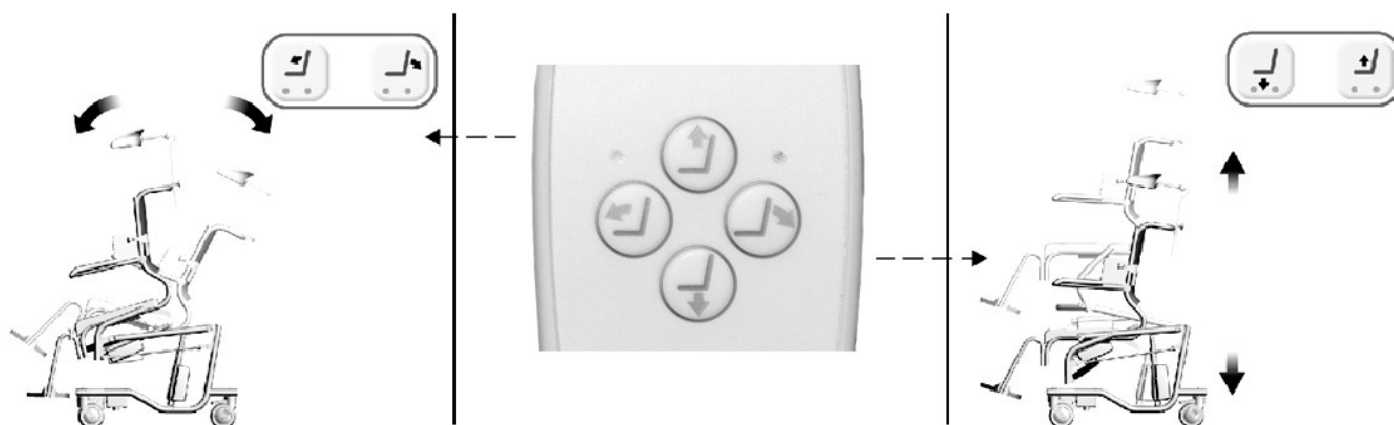EN: Diodes on the hand control - NB! 1. All diodes (1-6) flash - CAUTION - Error! Reset the chair if possible (see page 18). If it runs crooked - stop and seek technical help. 2. If there is light in diode (5) and (6), then the stop button is activated (see page 13). 3. Charging with power plug connected. If the diode (4) lights solid, the chair is fully charged, if it blinks, it is not fully charged. 4. Power save mode. If the chair / hand control is not used, the diodes are switched off on the hand control.

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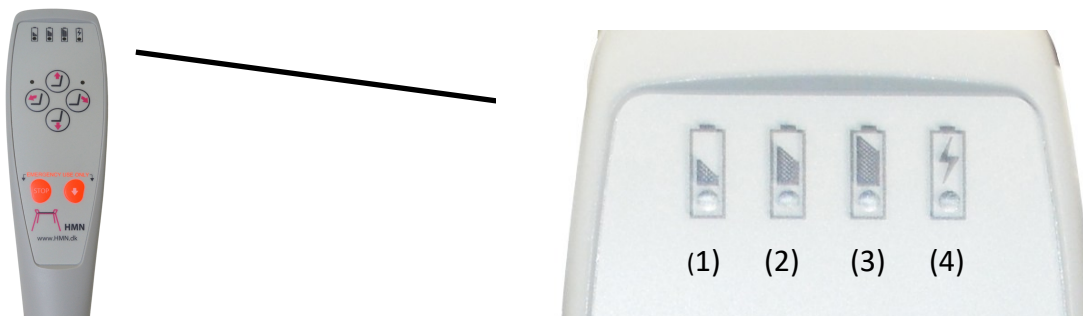
EN: It is possible to check the remaining power at the battery symbols (1, 2, 3) at the top of the hand control.
Remaining power: (1) = 10 - 20 %, (2) = 21 - 40 %, (3) = 41 - 100 %. When it reaches (2) or (1) the chair must be charged. When charging, the charging diode (4) lights continuously followed by flashing for a short period of time. A full charge takes approx. 12 hours. NB! The chair can't be used when charging.

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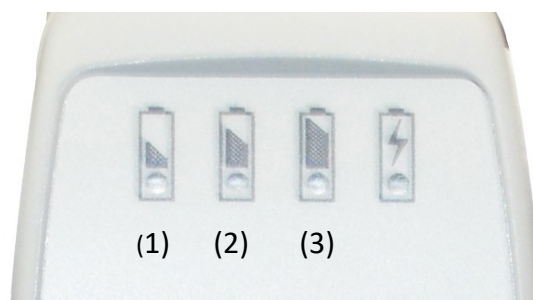
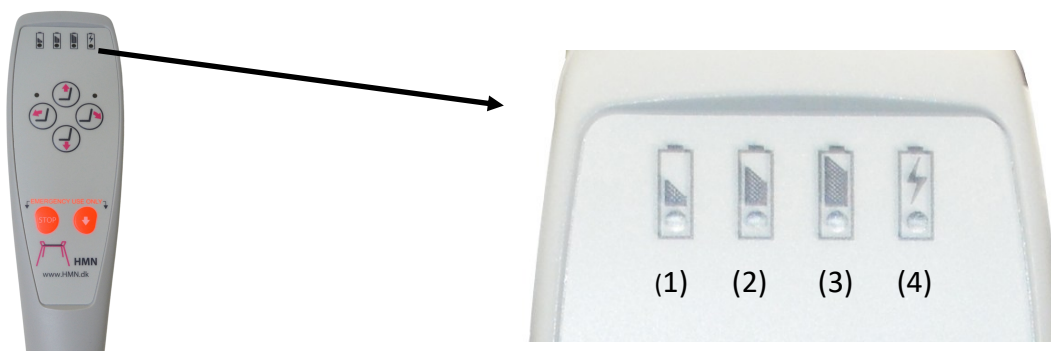
EN: About battery. Upon receipt of a new chair: The chair is fully charged in connection with production, but since there may be a power loss during storage / transport, the chair should not be expected to work before a full charge (= 12 hours or until the battery light stops lighting).

Regarding power loss. If the chair is not in use: At temperatures from about +10 degrees Celsius to +30 degrees Celsius: Expect a power loss per day of approx. 1½ to 2½ %. At temperatures below 10 degrees Celsius: Expect that the battery can be discharged in less than a day.

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EN: Resetting the system. If all the lights on the hand control flash, it may be due to a system fault. First hold down the "stop" button on the hand control until the two lamps at the arrow keys on the hand control light up (see page 13), then hold down the emergency button on the hand control until the chair is automatically lowered to the bottom. Now the chair works again. NB! It takes approx. 5 seconds before the chair is lowered, after it has "beeped" 5 times.

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EN: Attention! Not to be used in chlorine-containing environments as Swimming pools etc.

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EN: Cleaning

The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.

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EN: Complaint rights for frame

HMN grants 5 years complaint rights against frame break-down for all M2 Shower/commode chairs and stainless steel shower stools. The complaint rights applies for production and material break downs, and can only be used if the products have been in normal use. Please note that the normal wear and tear is not covered under complaint rights.

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Trouble Shooting/guide for electrical issues with M2 MULTI-TIP chairs.

NB! AS A FIRST COURSE OF ACTION CONCERNING ELECTRICAL ISSUES, ALWAYS TRY TO CHARGE THE CHAIR FOR

1. If only the left/last battery light is on

The chair is low on power and must be charged. The chair will not elevate if only 1 battery light is on.

2. If the 2 smaller lights at the arrow buttons on the hand control is on.

The emergency STOP has been activated. It must be deactivated by holding down the emergency stop button on the hand control until the chair beeps once and the 2 small diodes at the arrow keys go out.

3. All 4 lights at the top of the hand control is blinking.

The chair's control box has received an error signal. The chair must be reset. This is done by first pressing the emergency stop button on the hand control until the emergency stop is activated, and then holding the button next to the emergency stop button. First the chair will beep 3-4 times, then it will lower the chair completely. When the chair is stationary, it is reset.

If the chair after this is still blinking in all 4 top lights on the hand control it means there is a defect component.

Contact HMN for further advise.

4. If none of the lights on the hand control is on (after pressing any of the buttons), and the chair can't elevate or tilt.


Check that all the wires are correctly plugged in.

Try charging the chair for 24 hours after checking the wires.

If the problem continues after this it might be a defect battery or other electrical components.

Contact HMN for further advise and trouble shooting.

The picture on the left shows the control box:

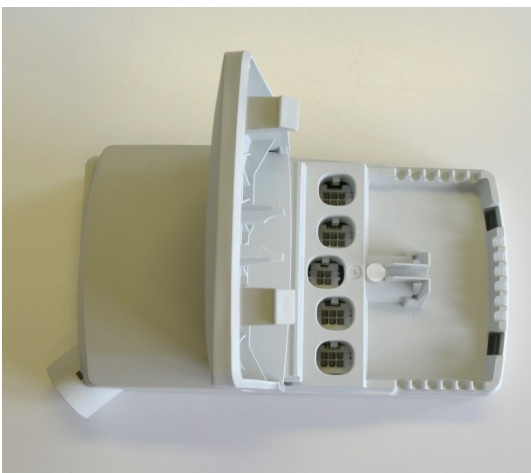Issues charging the chair.

1. If the chair wont charge after hours of trying.

Check that the outlet you are using has power in it, and that it is turned on.

2. The chair won't start charging, and the light indicating charging on the hand control is not blinking.

At the top and to the right on the hand control is a light indicating charging status. This light will blink if the battery is charging. At first when you plug the wire for charging in to an outlet this light will not blink, it will just light up. After some time the light will start to blink, and this indicates that the chair is charging. The reason some time might pass before it starts charging is that the system is prepping the battery to receive power. This might take extra time if the battery is cold, or the chair is very low on power.

Try to reset the system as described above in the first passage. If the problem continues contact HMN for further advice.

3. The charging light on the hand control is not lighting up or blinking even if the chair is plugged in to an outlet.

Check that the charging wire connected to the control box is correctly plugged in. Check that the red locking ring is mounted inside the plug on the control box. This issue is normally because the charging wire or the locking ring is incorrectly mounted. It can also be a faulty battery.

If the red locking ring is mounted correctly it can be seen through the plug like on the picture to the left:

Periodic errors.

1. The chair won't charge every attempt.

If experiencing periodic errors always try and reset the chair as described above. It is also possible that the issue is an incorrectly mounted wire, so check the wires is plugged correctly in. If the problem continues it might be a defect switch box. Contact HMN for further advice.

2. The chair will only work periodically, even after a reset.

If you often experience issues with the chair not working and the resets only work temporarily before all lights in the hand control are blinking again it might be sign of incorrectly mounted wires or issues with the switch box. Contact HMN for further advice.

The chair is elevating uneven, one defect actuator.

1. If one of the actuators on the chair isn't elevating and the chair is being pulled sideways.

The actuators on the chair will not always go at the same speed. This might result in slightly tilted seat frame. In this case just lower the chair completely to reset the pistons in the actuators.

If there is a problem with one actuator not working it might pull the chair sideways, and this indicates there could be a problem with that actuator.

Do NOT try to elevate the chair if this is the case. With this issue we need to know if the frame has been pulled out of shape. Send us pictures of the frame from the front and back, and we will be able to tell if the frame is damaged.

If not, just changing the actuator is enough, but if there is damage to the frame, we might need the chair sent to us for repair, or will send you the needed parts to replace. With these issues always contact HMN.

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To ensure that the aid functions safely for the user, we recommend that the following inspections be carried out.

1. Check that the frame is without breaks and the chair is intact.
2. Check that the backrest is correctly mounted with 2 screws on each side.
3. Check that the wheels on the chair can roll freely.
4. If the chair has a directional wheel, check that it functions normally.
5. Check that the brakes on the wheels work.
6. Check that the seat is intact and there are no breaks in the PU foam.
7. Check that the armrests are correctly mounted, function, and there are no breaks in the PU foam.
8. Check that the back in netweave is correctly mounted and intact.
9. Check that the chair's tilt function works.
10. Check that the chair's electric lift works.
11. Check that the battery can be charged (see any user manual).
12. Check that the chair's emergency lowering and emergency stop work (see any user manual).

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EN:

This product should be recycled following national regulation. After disassembly of the product, sort the components based on what they are made of and recycle them individually. Please contact your local recycling place for guidance if necessary.

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