


USERS GUIDE TOILET AID MODEL FRIGG


Stool made to sit on when using the toilet.

Height is adjustable as follows:

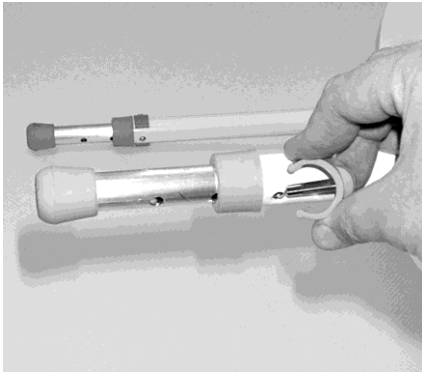

- 1) LAY DOWN THE STOOL ON THE SIDE.
- 2) REMOVE THE CLIPS.
- 3) THE LEG IS MOVED TO THE DESIRED HEIGHT
- 4) INSERT THE CLIPS AGAIN (NB! THE CLIPS MUST BE PUT THROUGH TO THE BOTTOM BEFORE USE).
- 5) THE SAME PROCEDURE IS REPEATED ON THE OTHER 3 LEGS.
- 6) IF NEEDED, THE CHAIR CAN BE TILTED BY MOVING THE REAR LEGS INTO **ONE POSITION** HIGHER THAN THE FRONT LEGS.

THE SPLASH GUARD OR BUCKET IS PUSHED IN THE INSIDE OF THE FRAME. THE TWO MARKS IN THE SPLASH GUARD OR BUCKET, FITS INTO THE MIDDLE OF THE FRAME.

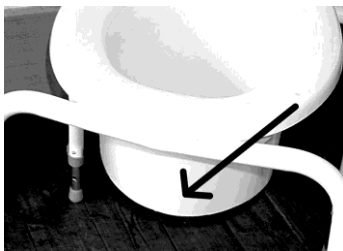

←ADJUSTING THE SPLASH GUARD

If the Splash guard is too deep for the toilet, it can be cut to the desired length with a knife.

Art. No. 610010

Max. user load 140 kg

NB! The clips have to be put all the way to the bottom of the holes before use.

2 year's complaint right against manufacturing errors.

Cleaning:

The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes



This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.

09.10.15

Artcl.no.:	SPARE PARTS::		
Freestanding toilet aid, model FRIGG.			
310065	Clips for height adjustment, grey		
310165	Clips for height adjustment, magenta		
610021	Splash guard		
610022	Bucket		
610020	Toiletseat, white		
610023	Strap (to hold seat)		
610024	Black rubber to hold the strap		
310075	Black rubber (to end of armrests)		
312033	Telescopic leg (unsprayed aluminium)		
310017	Rubbe shoes for telescopic stool		
310021	Bush for telescopic legs		

HMN a/s

Midtager 22, 2605 Brøndby, Danmark

Tel.: +45 43 96 24 00 Fax: +45 43 96 26 92 E-mail: info@hmn.dk

www.hmn.dk