

# USERS GUIDE: M2-SHOWER/COMMODE CHAIR WITH HIPSEAT.

Accessory: The seat is designed to use if you have the need for extra tilt at the back of the seat.

Page 1/2



**ADJUSTING THE SEAT HEIGHT AT THE BACK:  
THE TWO FINGER SCREWS ARE EASILY LOOSENED. THE SEAT IS KEPT IN  
THE POSITION YOU WANT AND THE TWO SCREWS ARE EASILY TIGHTENED.  
IN THE TOP POSITION THE BACK PART OF THE SEAT IS LIFTET APP. 8 CM  
COMPARED TO THE NORMAL POSITION. THAT IS  
APP. 11°.**

**NB! THE ADJUSTMENT MUST BE DONE WITHOUT THE USER IN THE CHAIR!**

## WARNING



DO NOT SIT ON THE OUTER 10 CM  
OF THE SEAT THAT THE CHAIR CAN  
TIP FORWARD.



DO NOT STAND ON THE  
FOODRESTS



Article. No. 310250/310251

**Max. User load 125 kg**

2 year's complaint right against manufacturing errors.

Cleaning: The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.





This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.

09.10.15

# MOUNTING INSTRUCTIONS: HIPSEAT FOR M2-SHOWER/COMMODE CHAIR

Accessory: The seat is designed to use if you have the need for an extra lift at the back of the seat.

Page 2/2

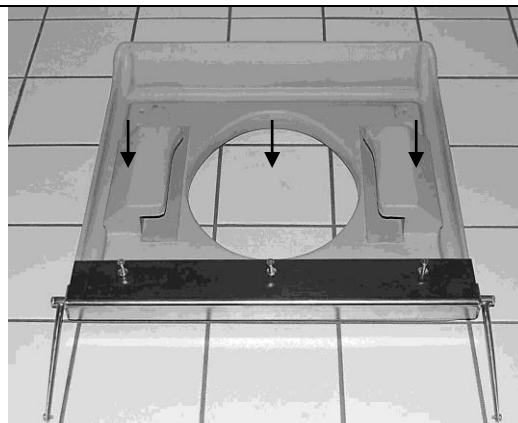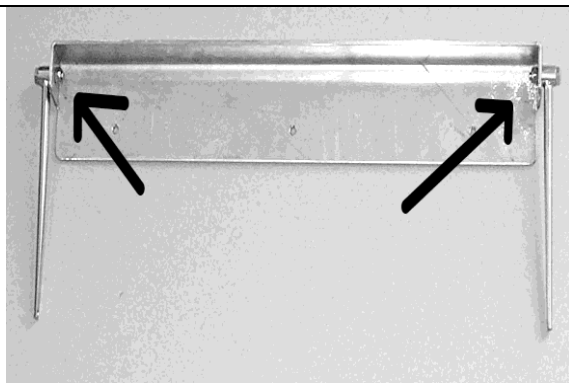


## 1). MOUNTING OF BARS FOR ADJUSTING THE HEIGHT.

Before the metal tray is screwed tight to the seat, the two bars for adjusting height (one on each side) is screwed to the metal tray with the enclosed bolts and locking-nuts which are placed on the inside.


## 2). THE METALTRAY IS FASTENED TO THE SEAT.

The seat is placed upside down. The metal tray is attached to the back of the seat. It is tightened by means of the three enclosed wooden screws (Inside the pu-foam is a wooden plate. It is in that the mounting is done).

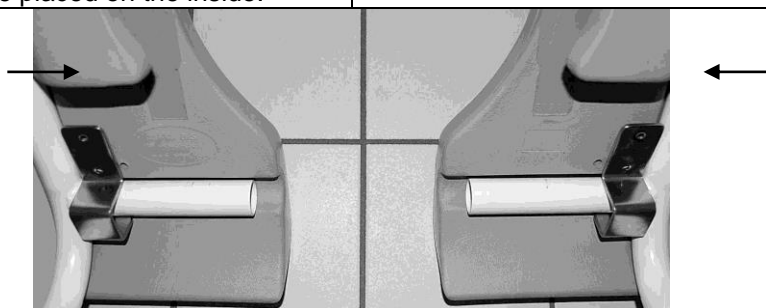

## 3). FITTINGS FOR THE FRONT BARS ARE MOUNTED.

The seat is placed upside down. The chair frame is placed on top. The two fittings are put in place and screwed on with the last four wooden screws. The chair is turned around and is now ready for completion.

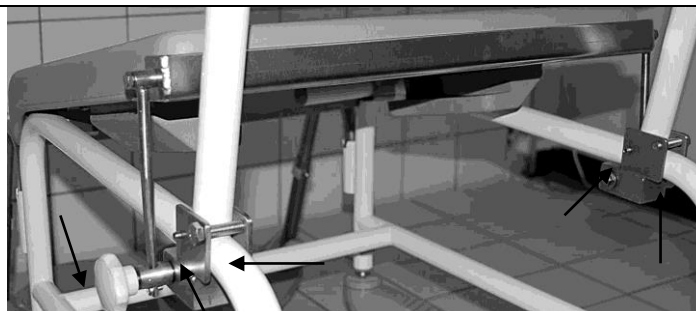

## 4). THE MOUNTING SET IS FASTENED TO THE CHAIR

The centre screws are firmly fastened to the frame. The fastening bolts are screwed in. Finger screws screwed in. The counter screws screwed firmly into the bars for adjusting the height

### Spare Parts

Art. No.	Description	Art. No.	Description
800027	Grey thumbwheel	800021	Pinol screws (small mini screws without head) 2pcs.
310081	Thumbwheel	800022	Fastening nuts, 2 pcs.
800019	Height adjustment bars, 2 pcs.	800023	Counter screws, 2 pcs.
800020	Bolt+locking nuts for height adj. Bars, 2 sets	310065	Clips for height adjustment, grey
800024	Wood screws, 7 pcs.	310165	Clips for height adjustment, magenta

HMN a/s

Midtager 22, 2605 Brøndby, Denmark

Tel.: +45 43 96 24 00 Fax: +45 43 96 26 92 E-mail info@hmn.dk

[www.hmn.dk](http://www.hmn.dk)