

User guide

M2 200 kg shower/commode chair

Locking the wheel



Before you use the chair for bathing/showering, you must make sure you apply the brake on all four wheels.

Directional controlled wheels



The right front wheel can be locked the direction. Use of this makes it easier to start and moving the chair.

Footrests



Pushing the footrests aside:
Lift them and turn them to the side you want. The plastic plate can be tipped up if you want to take them to take up very little space.

Height adjusting the footrests:
Adjust the height of the footrests by taking the clips out, placing the footrests in the wished position, and last put the clips back again.

Cleaning



To make the cleaning task easier you can click the seat on and off.

Armrests




To disassemble the armrest you have to fit the head of the bolt to where the armrest is milled out and pull it off.

How to put the seat on



Make sure that the back edge of the seat is fitted around the flat metal bar at the back.

Height adjustment

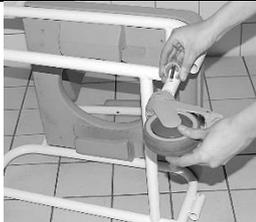

1. Lock all four wheels (by pressing down the grey plate on the wheels).
2. Lay down the chair.
3. Take the clips out of the leg, which is to be adjusted.
4. Take hold of the wheel and move it until the leg fits with a hole for the right height.
5. All 4 wheels should be placed in the same height.
6. Put the clips back again and repeat the procedure with the other wheels. (Turn the chair over to adjust the height on the last two wheels).


Removal of bucket



When sliding the bucket of the chair, it is important to slide it straight backwards, along the seat.

Otherwise it can be very difficult to get the bucket of the chair.

WARNING

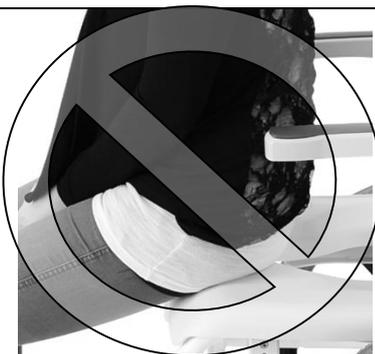

DO NOT SIT ON THE OUTER 10 CM OF THE SEAT THAT THE CHAIR CAN TIP FORWARD!



DO NOT STAND ON THE FOOTRESTS



Check with proper intervals the fastening of the wheels to the legs.

Art. No.: 310272

Max. Load: 200 kg

Attention: Push the Clips to the bottom when adjusting heights.

2 Years complaint rights against manufacturing errors.

Terms of claim for frame.

HMN grants 5 years claim rights against frame break-down for all M2 Shower/commode chairs and stainless steel shower stools.

The warranty applies for production and material break downs, and can only be used if the products have been in normal use.

Please note that the normal wear is not included in the warranty.

Cleaning:

The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.



This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.

09.10.15

HMN a/s

Midtager 22, 2605 Brøndby, Denmark

Tel.: +45 43 96 24 00 Fax. +45 43 96 26 92 E-mail info@hmn.dk

www.hmn.dk