


User guide

M2 shower/commode chair with 24 inch wheels

How to mount the wheels:

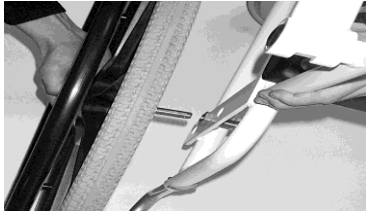

The axel is clicked into the bush. Pull the wheel to make sure it is correctly mounted.

But be careful not to tighten the bolts too much. Using the chair without the driving belt, the wheels will get more elastic.

(when you receive the chair it is adjusted to the middle position in height)



How to adjust the height:

The bushings to the 24-inch wheels are taken off with a wrench key and moved to the desired height.

The anti-tip bar is moved to the same position as the 24-inch wheels. The clips holding the front wheels are taken out, and the front wheels are moved to the desired height.


The front wheels can be pu in a position which is **ONE** (1) position different from the 25 inch wheels (if you want the chair to be slightly tilted)

All other adjustments shall be in the same position! (For example: in the top hole). Double check that all nuts and bolts are tightened and that all the clips are pressed in place.

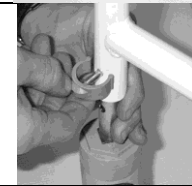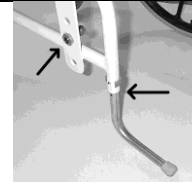

Make sure to tighten the bolts well


Mount the brakes

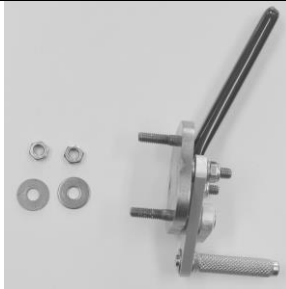

For mounting:
2 x Brakes, 4 x washers and 4 x bolts



Mount the brake on the chair.

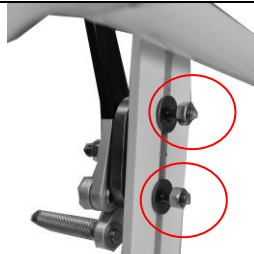

Mount the washers and bolts



Fasten the bolts



Adjusting the brakes

Loosen the bolts and move the brake so it has the wanted impact.





Tighten the bolts well!

Pan / bucket



When sliding the bucket of the chair, it is important to slide it straight backwards, along the seat.

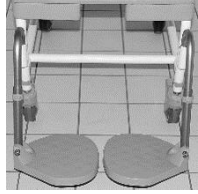Otherwise, it can be very difficult to get the bucket of the chair.

Footrests		
<p>Pushing the footrests aside: Lift them and turn them to the side you want. The plastic plate can be tipped up if you want to take them to take up very little space.</p> <p>Height adjusting the footrests: Adjust the height of the footrests by taking the clips out, placing the footrests in the wished position, and last put the clips back again.</p>		
 <p>WARNING</p> <p>RISK OF ROLLING OR GLIDE-OUT!</p> <p>TO AVOID THE CHAIR CAN ROLLS OR GLIDER IN TRANSFER, TO ALL 4 WHEELS BE LOCKED! ATTENTION!! IF THERE IS A SMOOTH SURFACE, THERE IS STILL A RISK OF THE CHAIR ROLLING OR GLIDING.</p>		
	<p>DO NOT SIT ON THE OUTER 10 CM OF THE SEAT THAT THE CHAIR CAN TIP FORWARD.</p> <p>←</p>	
<p>DO NOT STAND ON THE FOOTRESTS</p> <p>⇒</p>		
<p>Check with proper intervals the fastening of the wheels to the leg.</p>		
<p>2 Years complaint rights against manufacturing errors. Terms of claim for frame. HMN grants 5 years claim rights against frame break-down for all M2 Shower/commode chairs and stainless steel shower stools.</p> <p>The warranty applies for production and material break downs, and can only be used if the products have been in normal use.</p> <p>Please note that the normal wear is not included in the warranty.</p>	<p>Cleaning: The product can be cleaned with regular cleaning products without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.</p>	
<p>CE</p> <p>This product is CE marked which means the product complies with EU legislations and enables the free movement of the product in the European market.</p> <p>09.10.15</p>	<p>HMN a/s Midtager 22, 2605 Brøndby, Denmark Tel.: +45 43 96 24 00 Fax. +45 43 96 26 92 E-mail info@hmn.dk www.hmn.dk</p>	