

**USERS GUIDE: M2 300 kg and 300 kg wide SHOWER/COMMODE CHAIR.
Made for using when bathing/showering and going to the toilet.**

LOCKING THE WHEEL:



Directional controlling wheels



Before you use the chair for bathing/showering you must make sure you apply the brake on all four wheels.

The right front wheel can be locked in the direction. Use of this makes it easier to start and move the chair.

FOOTRESTS:



The footrests are mounted by placing them in the footrest holders.

CLEANING:



To make the cleaning task easier you can click the seat on and off.

DISASSEMBLY OF ARMRESTS



Disassembly of armrests by pulls up the armrest to 45 degrees, and pulls out the armrest.

TO PUT SEAT ON



Make sure that the back edge of the seat is fitted around the flat metal bar at the back.

Use of bucket



When sliding the bucket of the chair, it is important to slide it straight backwards, along the seat. Otherwise it can be very difficult to get the bucket of the chair.

WARNING



DO NOT SIT ON THE OUTER 10 CM OF THE SEAT THAT THE CHAIR CAN TIP FORWARD.



DO NOT STAND ON THE FOOTRESTS



Check with proper intervals the fastening of the wheels to the legs.

Artcl. No. 310280, 310284

Maximum user load 300 kg.

2 year's complaint right against manufacturing errors.

Terms of claim for frame.

HMN grants 5 years claim rights against frame break-down for all M2-Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that normal wear is not included in the warranty.

Cleaning: The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.

| | |
|-------------|-------------------------------------|
| Article No: | (GB) Spare parts |
| 310354 | Intimate seat for M2 300 kg's, grey |
| | |
| | |
| | |



This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.

09.10.15

HMN a/s
Midtager 22, 2605 Brøndby, Denmark
Tel.: +45 43 96 24 00 Fax.: +45 43 96 26 92
E-mail info@hmn.dk www.hmn.dk