


**USER GUIDE HEIGHT ADJUSTABLE SHOWER/COMMUNE CHAIR, MODEL M2 BOXED CHAIR
INTENDED FOR USE A BATHING, SHOWERING AND TOILET VISIT.**

STEP 1. ASSEMBLY OF WHEELS



LAY DOWN THE CHAIR.
TAKE THE CLIPS OUT OF THE LEG AND INSERT
THE WHEELS INTO THE FRAME.
MAKE SURE THAT ALL 4 WHEELS ARE IN THE
SAME HEIGHT BEFORE INSERTING THE CLIPS.
TURN THE CHAIR OVER FOR EASIER
ASSEMBLY OF THE WHEELS.

STEP 2. ASSEMBLY OF BACK ON FRAME

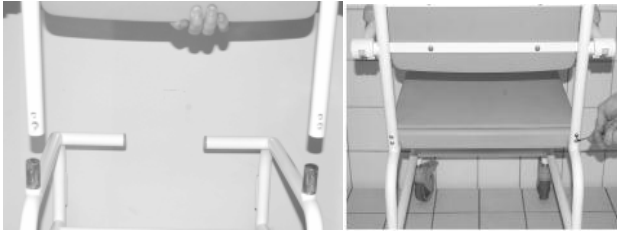


ATTACH THE BACK INTO THE FRAME.
INSERT THE 4 SCREWS AND TIGHTEN WITH AN
ALLEN WRENCH.
MAKE SURE THAT ALL 4 SCREWS ARE
SECURELY TIGHTENED.
REMEMBER TO RETIGHTENING AFTER A
MONTH.
THE SCREWS SHALL BE DISCARDED AFTER
THE MOUNTING AND YOU SHALL USE THE
SCREWS TO THE NEXT MOUNTING.
Art. No. 800290

STEP 3. ASSEMBLY/DISASSEMBLY OF ARMREST



TO ATTACH OR DETACH THE ARMRESTS, HOLD
THE ARMRESTS IN A 45 DEGREE ANGLE AND
FIT IT OVER THE ARMREST HOLDER.

STEP 4. LOCKING OF WHEELS

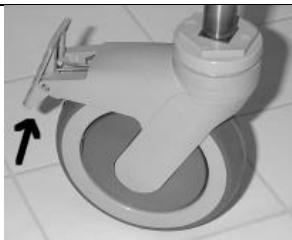

BEFORE USE OF THE CHAIR BATHING,
SHOWERING OR LIFTING PERSONS INTO THE
CHAIR, MAKE SURE THAT ALL 4 WHEELS ARE
LOCKED.
TO LOCK THE CHAIR, PRESS DOWN THE GREY
PLATE AS SHOWN ON THE PICTURE TO THE
LEFT.

MOUNTING OF FOOTRESTS



ADJUST THE HEIGHT OF THE FOOTRESTS BY
TAKING THE CLIPS OUT, PLACING THE
FOOTRESTS IN THE WISHED POSITION AND
PUTTING THE CLIPS BACK IN.
FOOTRESTS ARE TO BE PLACED ON THE
FRONT OF THE FRAME IN THE FOOTRESTS
HOLDERS.

THE PAN/BUCKET



WHEN SLIDING THE BUCKET OF THE CHAIR, IT
IS IMPORTANT TO SLIDE IT STRAIGHT
BACKWARDS ALONG THE SEAT.
OTHERWISE IT CAN BE VERY DIFFICULT TO
GET THE BUCKET OF THE CHAIR.

WARNING



DO NOT SIT ON THE OUTER 10
CM OF THE SEAT THAT THE
CHAIR CAN TIP FORWARD.



DO NOT STAND ON THE
FOODRESTS



Art. No.: 310700/ 310701

2 Years complaint rights against manufacturing errors.

Terms of claim for frame.

HMN grants 5 years claim rights against frame break-down for all M2 Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that the normal wear is not included in the warranty.

Cleaning: The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.

Artkl. Nr.: Spare parts

310340	Standard seat, grey
310342	Seat with extra large hole, grey
310341	Seat lid for std. seat, grey
310343	Seat lid for seat with extra large hole, grey
310065	Clip for height adjustment, long, grey
310165	Clip for height adjustment, long, Magenta
800181	Pu foam padding for straight armrest, 1 pcs. Grey
800241	Pu foam padding for curved armrest, 1 pcs. Grey
800012	Screws for fastening of pu foam padding on armrest, 2 pcs.
310076	White nylon bush for armrests
800082	Bolt for armrests
800013	White nylon bush for telescopic leg with wheel
310344	Pu foam back-rest, grey
310063	Screws for fastening of back-rest, 4pcs.
800290	Screws for assembling of frame, 4 pcs.
800244	Grey ferule for M2 with curved pushing frame, round armrest
310066	Armrest set for straight up, set with 2 pcs.
310067	Armrest set with pu foam for straight up, set with 2 pcs., Black
310064	100 mm wheel with leg on, 1 pcs.
800042	100 mm wheel with leg on, set with 4 pcs.
310073	Black plastic ferrule for flat armrests, black
800245	Grey ferrule for oval armrest
800025	Telescopic leg, 1 pcs.



This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.

09.10.15

HMN a/s
Midtager 22, 2605 Brøndby, Denmark
Tel.: +45 43 96 24 00 Fax. +45 43 96 26 92 E-mail
info@hmn.dk
www.hmn.dk