

IMPORTANT!
**WHEN CHARGING, DO NOT TURN ON THE SWITCH
BEFORE THE CHARGER IS CONNECTED TO THE
CHAIR.**

**THE BATTERY SHOULD BE CHARGED 24 HOURS THE
FIRST TIME.**

**WHEN THE BATTERY HAS RUN DOWN THERE IS A
BEEP SOUND FROM THE CONTROL BOX WHEN THE
ELECTRICAL PARTS ARE IN USE.**

**WHEN THE BEEP SOUND COMES, YOU HAVE TO
RECHARGE THE BATTERY AS IT CAN BE DAMAGED IF
YOU LET IT RUN 100 % OUT OF POWER.**

**(IF THE BATTERY HAS RUN OUT OF POWER, YOU
CANNOT USE THE ENGINE BY PUTTING THE
CHARGER IN THE WALL PLUG AND THE CHAIR.
A FULL RECHARGE TAKES TYPICALLY 8 HOURS.)**

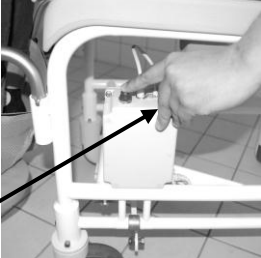
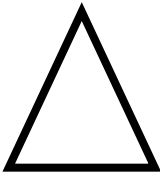
**IF THE LIGHTBOLT ON THE CHARGER LIGHTS GREEN
WHEN CONNECTED, IT IS DUE TO A MALFUNCTION
OF THE BATTERY, AND THEREFORE IT NEEDS TO BE
CHANGED.**

**DO NOT CHARGE THE CHAIR WHILE BEING USED IN
AREAS WITH WATER.**



**DO NOT USE THE ELECTRICAL FUNCTIONS ON THE
CHAIR WHILE SPRAYING WITH WATER.**

USERS GUIDE: M2- SHOWER/COMMODE CHAIR.
 Made for using when bathing/showering and going to the toilet

LOCKING THE WHEEL:

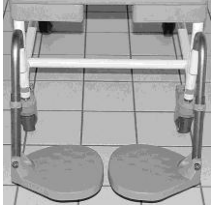
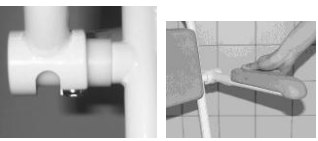
	<p>BEFORE YOU USE THE CHAIR FOR BATHING/SHOWERING YOU MUST MAKE SURE YOU APPLY THE BRAKE ON THE WHEELS. YOU BRAKE THE WHEELS WITH ONE TOUCH ON THE RED BUTTON, AND THE BUTTON SIT ON LEFT SIDE ON THE CHAIR</p>		<p>NOTICE: THE USER CAN REACH THE BUTTON AND UNLOCK THE WHEELS.</p>
---	---	--	--

CHARGING A BATTERY

	<p>DO NOT CHARGE THE CHAIR WHILE BEING USED IN AREAS WITH WATER. YOU GRAB AROUND THE SOCKET ON THE CHAIR AND, YOU CAN PULL THE PLUG FROM THE TRANSFORMER PLACE.</p>		<p>YOU GRAB AROUND THE CHARGING PLUG (SOCKET) ON THE CHAIR AND YOU PUSH THE PLUG FROM THE TRANSFORMER IN PLACE. NOW IT IS READY FOR CHARGING</p> <p>OBS! PLEASE NOTE THAT THE PLUG MUST BE CLOSED, AFTER EVERY CHARGING.</p>
---	---	--	---

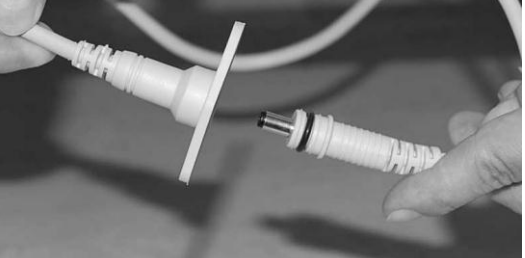
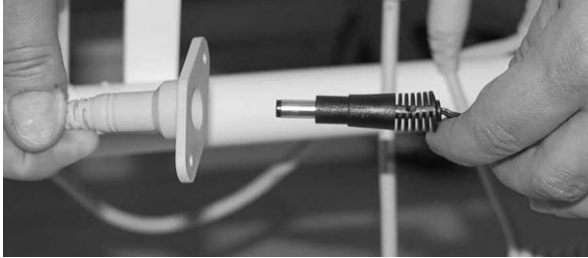
USE OF FOOT RESTS

RELEASING THE ARMREST:

	<p>MOUNTING OF FOOT RESTS ON THE CHAIR THE HEIGHT OF THE FOOT RESTS ARE ADJUSTED BY TAKING THE CLIPS OUT AND SUBSEQUENTLY PLACING THE FOOT RESTS IN A SUITABLE HEIGHT AND THEN REINSERTING THE CLIP. PLACING THE FOOT RESTS ON ONE SIDE: YOU LIFT THE FOOTRESTS SLIGHTLY AND SWING THEM TO EITHER SIDE YOU WANT. IF YOU WANT THE FOOT RESTS TO OCCUPY LESSER SPACE YOU ARE ABLE TO TILT THEM.</p>		<p>← PULL THE ARMREST UP TO 45 DEGREES AND THEN YOU CAN TILT THE ARMREST.</p>
--	---	---	---

1) CHARGING THE BATTERY

2) CHARGING THE BATTERY



	
---	--

WITH THE ONE HAND, YOU GRAB AROUND THE CHARGING PLUG (SOCKET) ON THE CHAIR AND WITH THE OTHER HAND YOU PULL THE CONNECTING CABLE FOR THE BATTERY.

WITH THE ONE HAND, YOU GRAB AROUND THE CHARGING PLUG (SOCKET) ON THE CHAIR AND WITH THE OTHER HAND, YOU PUSH THE PLUG FROM THE TRANSFORMER IN PLACE.

3) CHARGING THE BATTERY

USE OF BUCKET:

	<p>DEPENDING ON REGION PLUG TYPE 1, 2 OR 3 SLIDES INTO THE CHARGER. AFTERWARDS YOU PUT THE TRANSFORMER IN THE SOCKET AND TURN ON THE POWER. A FULL CHARGE TAKES APPROXIMATELY 8 HOURS. WHEN THE BATTERY IS RUNNING OUT OF POWER YOU WILL HEAR A "BEEP" SOUND.</p> <p>WHEN THE BATTERY IS BEING CHARGED, THE LIGHT BOLT ON THE CHARGER LIGHTS ORANGE. WHEN THE BATTERY HAS BEEN FULLY CHARGED, THE LIGHT BOLT ON THE BATTERY WILL LIGHT GREEN.</p> <p>NOTE: IF THE LIGHT BOLT ON THE CHARGER LIGHTS GREEN, WHEN IT IS BEING CONNECTED, IT IS DUE TO A MALFUNCTION, AND IT NEEDS TO BE CHANGED.</p>		<p>WHEN SLIDING THE BUCKET OF THE CHAIR, IT IS IMPORTANT TO SLIDE IT STRAIGHT BACKWARDS, ALONG THE SEAT. OTHERWISE IT CAN BE VERY DIFFICULT TO GET THE BUCKET OF THE CHAIR.</p>
---	---	---	---

Artikel. Nr. 310208/310209

**Maximum user load 150 kg.
2 year's complaint right against manufacturing errors.**

Terms of claim for frame.

HMN grants 5 years claim rights against frame break-down for all M2-Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that normal wear is not included in the warranty.

Cleaning: The product is produced in stainless materials and can be cleaned with regular cleaning products without grinding and corroding effect. We advise against using solvents and chlorine. Polyurethan can be washed up to 80 degrees C and can be cleaned with ethyl and propyl alcohol up to dissolution of 70 %. Wash and dry the product after cleaning.

Artkl. Nr.:	Spareparts:
310340	Standard seat, grey
310342	With extra large Seat hole, grey
310344	Back rest (pu foam), grey
310063	Screws for fastening of back rest, 4 pcs.
310066	Armrests for straight up, set with 2 pieces
310067	Armrests for straight up with pu foam pillows, set w/ 2 pieces
800181	Pu foam pillow for armrests, straight armrests, 1 pcs., grey
800012	Screw for fastening of armrest pillows.
310076	Nylon bush for armrests
800082	Bolt for armrest
800244	Grey plastic shoe for round armrests, grey
800245	Grey plastic shoe for flat oval armrests, grey



22.10.12 Produced in the EU

HMN a/s
Midtager 22, 2605 Brøndby, Denmark
Tlf.: +45 43 96 24 00 Fax.: +45 43 96 26 92 E-mail info@hmn.dk
www.hmn.dk