

**IMPORTANT!**  
**WHEN CHARGING, DO NOT TURN ON THE SWITCH  
BEFORE THE CHARGER IS CONNECTED TO THE  
CHAIR.**

**THE BATTERY SHOULD BE CHARGED 24 HOURS  
THE FIRST TIME.**

**WHEN THE BATTERY HAS RUN DOWN THERE IS A  
BEEP SOUND FROM THE CONTROL BOX WHEN THE  
ELECTRICAL PARTS ARE IN USE.**

**WHEN THE BEEP SOUND COMES, YOU HAVE TO  
RECHARGE THE BATTERY AS IT CAN BE DAMAGED  
IF YOU LET IT RUN 100 % OUT OF POWER.**







**(IF THE BATTERY HAS RUN OUT OF POWER, YOU  
CANNOT USE THE ENGINE BY PUTTING THE  
CHARGER IN THE WALL PLUG AND THE CHAIR.  
A FULL RECHARGE TAKES TYPICALLY 8 HOURS.)**

**IF THE LIGHTBOLT ON THE CHARGER LIGHTS  
GREEN WHEN CONNECTED, IT IS DUE TO A  
MALFUNCTION OF THE BATTERY, AND THEREFORE  
IT NEEDS TO BE CHANGED.**



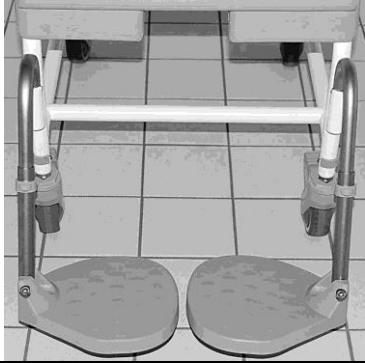


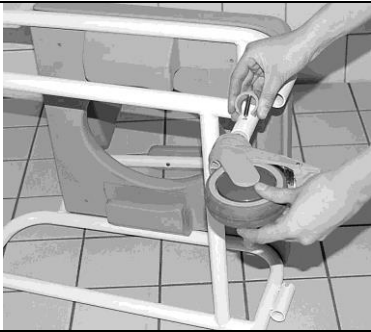
**DO NOT CHARGE THE CHAIR WHILE BEING USED IN  
AREAS WITH WATER.**

**DO NOT USE THE ELECTRICAL FUNCTIONS ON  
THE CHAIR WHILE SPRAYING WITH WATER.**

User guide for Shower/Commode chair M2 EI-Tip.  
 Intended for use a bathing, showering and toilet visit.  
 (2/6)


	<p><b>Assembly of wheels:</b></p> <p>Lay down the chair. Take the clips out of the leg and insert the wheels into the frame. Make sure that <u>all</u> 4 wheels are in the same height position before inserting the clips.        Turn the chair over for easier assembly of the wheels.</p>
<p><b>Mounting the back:</b></p> <p>Attach the back into the frame.</p>	
	<p><b>Assembly the back:</b></p> <p>Insert the 4 screws and tighten with an Allen wrench. Make sure that <u>all</u> 4 screws are securely tightened.        Remember to retighten the screws after a month. The screw shall be discarded after the mounting and you shall use the screws to the next mounting.        (Art. No 800290)</p>
<p><b>Mounting/Disassembly of armrest</b></p> <p>Pull the snap release and then you can tilt the armrest.</p>	
<p><b>Locking the wheel</b></p>  <p>Before you use the chair for bathing/showering, you must make sure you apply the brake on all four wheels.</p>	<p><b>Directional controlled wheel</b></p>  <p>The right front wheel can be locked the direction. Use of this makes it easier to start and moving the chair.</p>
<p style="text-align: center;">Continued on page 3</p>	

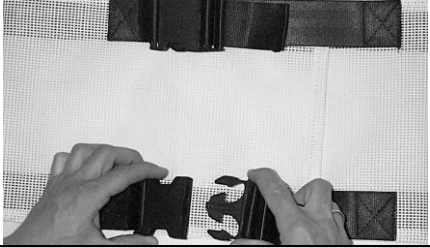**User guide for Shower/Commode chair M2 EI-Tip.**  
 Intended for use a bathing, showering and toilet visit.  
 (3/6)
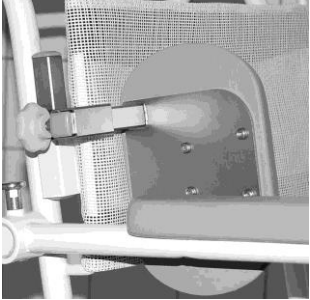
<b>Back adjustment</b>	
	<p>You open the back by pressing the two plastic taps with your left hand, and then with your right hand you pull the buckle out.</p> <p>The depth of the back is adjusted by pulling the webbing and then tighten it.</p> <p>Care: Washable at maximum 80 degrees.</p>
	
<p align="center"><b>Mounting the footrests</b></p> <p>The height of the footrests are adjusted by taking the clips out and subsequently placing the footrests in a suitable height and then reinserting the clip. Placing the footrests on one side: You lift the footrests slightly and swing them to either side you want. If you want the footrests to occupy lesser space you are able to tilt them.</p>	
	<p align="center"><b>Calf support adjustment</b></p> <p>The calf support is fastened with velcro tape on the backside.</p> <p>The depth and the Calf support is adjusted by fastening the velcro tape further in or further out.</p> <p>Care: Washable at maximum 80 degrees.</p>
<p align="center"><b>The pan/bucket</b></p> <p>When sliding the bucket of the chair, it is important to slide it straight backwards, along the seat. Otherwise it can be very difficult to get the bucket of the chair.</p>	
	<p align="center"><b>Side support adjustment</b></p> <p>The side supporters are clicked into place in the square pipes on the side of the chair.</p> <p>The depth is adjusted by loosening the finger screw and then tightening it again once the height is right. You can also fasten the side supporters so that the cushions are turned horizontally if there is a need for that.</p>
Continued on page 4	

# User guide for Shower/Commode chair M2 EI-Tip. Intended for use a bathing, showering and toilet visit. (4/6)

## Adjusting the headrest (Art. No. 310221)

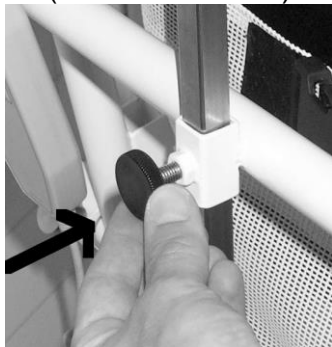

The height of the headrest is adjusted by loosening the finger screw and tightening it again when the height is right.

## Adjusting H+D headrest (Art. No. 310222)



The height and depth of the headrest is adjusted by loosening the two finger screws and then tightening them again once they are in the right position.

## Use of chair in negative

Use of chair in negative position can cause that the user will fall forwards.



## Tilting

For going up or down the chair is tilted by using the remote-control.

If the user is not able to hold on to the armrest by himself, the helper must lay the users hands on his lap before tilting the chair.

## Charging the battery

Depending on region plug type 1, 2 or 3 slides into the charger.

Afterwards you put the transformer in the socket and turn on the power.

A full charge takes approximately 8 Hours.

When the batteri is running out of power you will hear a "beep" sound.

When the battery is being charged, the light bolt on the charger lights orange.

When the battery has been fully charged, the light bolt on the battery will light green.


Note: If the light bolt on the charger lights green, when it is being connected, it is due to a malfunction, and it needs to be changed.



Continued on page 5


**User guide for Shower/Commode chair M2 EI-Tip.**  
 Intended for use a bathing, showering and toilet visit.  
 (5/6)

**Charging the battery**

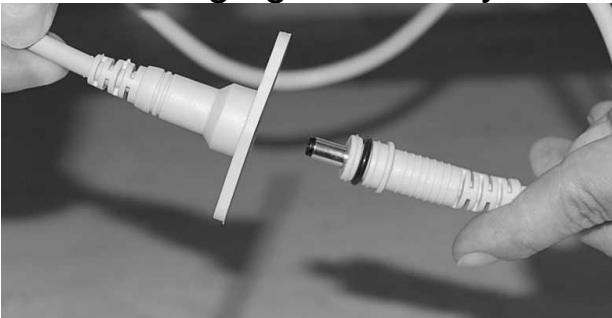


With the one hand, you grab around the charging plug (Socket) on the chair and with the other hand you pull out the connecting cable for the battery.

**Charging the battery**

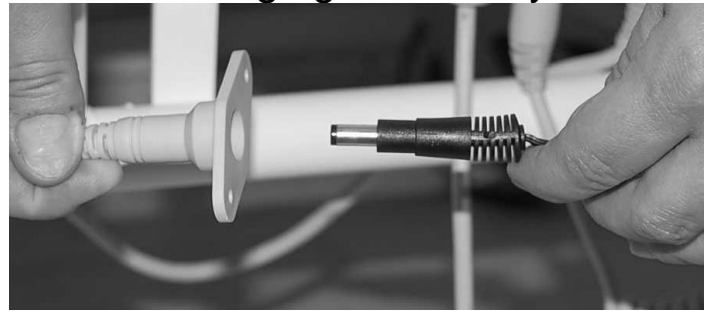

With the one hand, you grab around the charging plug (Socket) on the chair and with the other hand, you push the plug from the transformer in place.

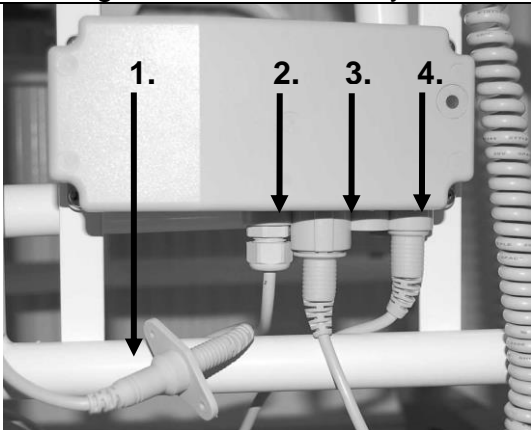

**Cables**

1. Plug for battery
2. Cable for battery
3. Cable for remote control
4. Cable for actuator ("engine")

**WARNING**



Do not sit on the outer 10 cm of the seat that the chair can tip forward.

←

Do not stand on the footrests.

→



Art. No:

**Standard seat:**

310210/310210-B, 310608/310608-B

**Max. Load 140 kg**

**Attention! Push the Clips to the bottom when adjusting heights.**

2 Years complaint rights against manufacturing errors.

**Terms of claim for frame.**

HMN grants 5 years claim rights against frame break-down for all M2 Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that the normal

**Seat with extra large hole:**

310211/310211-B, 310609/310609-B

**Cleaning:**

The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.

This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.



**See the spare parts on page 6**

User guide for Shower/Commode chair M2 EI-Tip.  
Intended for use a bathing, showering and toilet visit.  
(6/6)

Art. no	Spare Parts	Art. no	Spare Parts
310282	Back (Grey netweave)Chairs from Jan. 2011	310254	Quick release lock for armrests
310252	Back (White plastic net)Chairs up to Jan. 2011	800294	Bolt and mounting for fastening of frame
800326	Height, width adjustment bar	310276	Foot plate, right
800327	Screws for mounting of adjustment bar	310277	Foot plate, left
800546	Screws for mounting of adjustment bar for height, depth and width	800170	Clips, long
800191	Grey Headrest cushion and mounting for the depth bar	800169	Bolt for foot plate with nut
800328	Depth adjustment bar	310226	Leg support (White nylon with velcro)
800262	Vertical tube for headrest, height and depth adjustable (incl no. 7+8)	800233	Upper frame
800027	Finger screw, grey	800156	Bottom frame
800249	Grey ferule for headrest	800244	Ferule for round armrest, grey
310094	125 mm wheel with leg on. 1 pcs.	800012	Screws for fastening of armrest
800277	100 mm wheel with leg on. 1 pcs.	800082	Bolt for armrest
310065	Clips	<b>Electrical parts</b>	
800013	Bush, wheel	800267	Remote control (including wire and plug)
310340	Standardseat, grey	800269	Control box
310341	Seat lid for std. seat, grey	800270	Battery box incl. battery
310342	Seat with extra large hole, grey	800268	Actuator ("engine")
310343	Seat lid for seat with extra large hole, grey	800280	Transformer (Euro plug)
310352	Armrest for M2 with quick release & grey pu foam, right	800279	Transformer (British plug)
310352	Armrest for M2 with quick release & grey pu foam, left	800281	Transformer (US/Japan plug)
800181	Pu foam	800272	Bolt with nut for fastening of actuator (on top)
310076	White nylon bush for armrest	800271	Bolt for fastening of actuator (below bottom)

**HMN A/S**

Midtager 22, 2605 Brøndby, Denmark  
Tel. +45 43 96 24 00, Fax. +45 43 96 26 92  
E-mail. [info@hmn.dk](mailto:info@hmn.dk) Website. [www.hmn.dk](http://www.hmn.dk)