

IMPORTANT!
WHEN CHARGING, DO NOT TURN ON THE SWITCH
BEFORE THE CHARGER IS CONNECTED TO THE
CHAIR.

THE BATTERY SHOULD BE CHARGED 24 HOURS
THE FIRST TIME.

WHEN THE BATTERY HAS RUN DOWN THERE IS A
BEEP SOUND FROM THE CONTROL BOX WHEN THE
ELECTRICAL PARTS ARE IN USE.

WHEN THE BEEP SOUND COMES, YOU HAVE TO
RECHARGE THE BATTERY AS IT CAN BE DAMAGED
IF YOU LET IT RUN 100 % OUT OF POWER.

(IF THE BATTERY HAS RUN OUT OF POWER, YOU
CANNOT USE THE ENGINE BY PUTTING THE
CHARGER IN THE WALL PLUG AND THE CHAIR.
A FULL RECHARGE TAKES TYPICALLY 8 HOURS.)

IF THE LIGHTBOLT ON THE CHARGER LIGHTS
GREEN WHEN CONNECTED, IT IS DUE TO A
MALFUNCTION OF THE BATTERY, AND THEREFORE
IT NEEDS TO BE CHANGED.

DO NOT CHARGE THE CHAIR WHILE BEING USED IN
AREAS WITH WATER.

DO NOT USE THE ELECTRICAL FUNCTIONS ON
THE CHAIR WHILE SPRAYING WITH WATER.

USERS GUIDE: M2-FLEXI-TIP SHOWER/COMMODE CHAIR. (2/5)

Made for using when bathing/showering and going to the toilet.


TILTING:



TILTING:

The two top buttons are for tilting of the back rest. The lower buttons are for tilting of the seat. If the user is not able to hold on to the armrest by himself, the helper must lay the users hands on his lap before tilting the chair.

LOCKING THE WHEEL

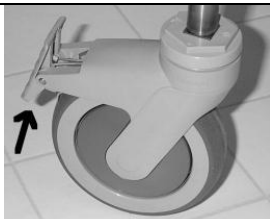


DIRECTIONAL CONTROLLED WHEELS




Before you use the chair for bathing/showering, you must make sure you apply the brake on all four wheels.

The right front wheel can be locked the direction. Use of this makes it easier to start and moving the chair.

1) CHARGING THE BATTERY

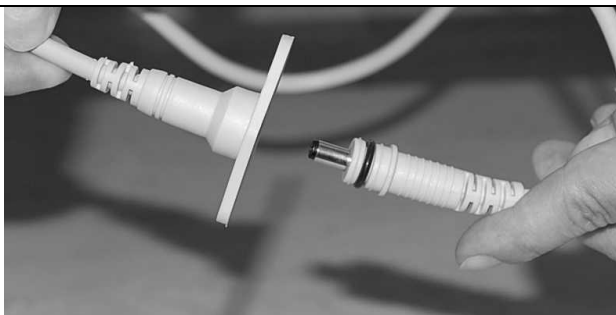


2) CHARGING THE BATTERY

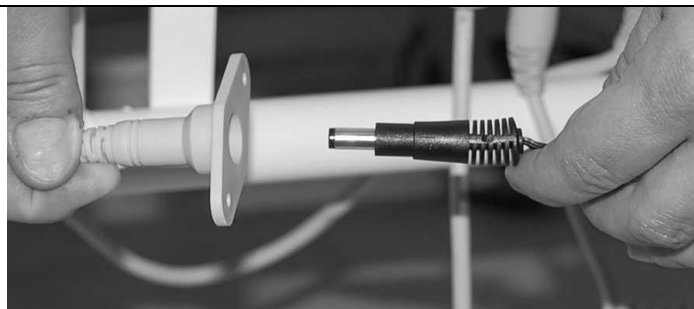

With the one hand, you grab around the charging plug (Socket) on the chair and with the other hand you pull out the connecting cable for the battery.

With the one hand, you grab around the charging plug (Socket) on the chair and with the other hand, you push the plug from the transformer in place.

3) CHARGING THE BATTERY

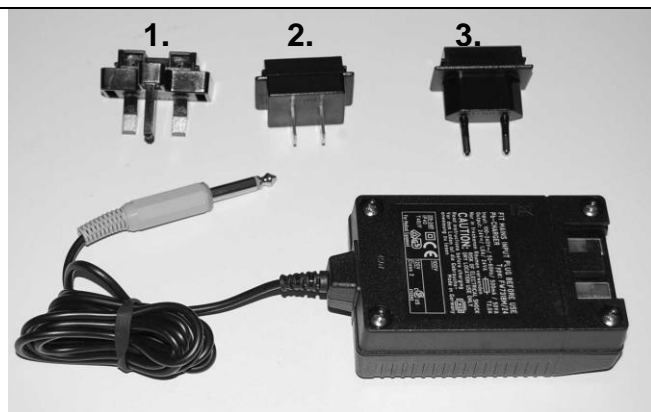

Depending on region plug type 1, 2 or 3 slides into the charger. Afterwards you put the transformer in the socket and turn on the power.

A full charge takes approximately 8 Hours.

When the batteri is running out of power you will hear a "bip" sound.

When the battery is being charged, the light bolt on the charger lights orange.


When the battery has been fully charged, the light bolt on the battery will light green.

Note: If the light bolt on the charger lights green, when it is being connected, it is due to a malfunction, and it needs to be changed.

USERS GUIDE: M2- FLEXI -TIP SHOWER/COMMODE CHAIR. (3/5)

Made for using when bathing/showering and going to the toilet.


4) CABLES

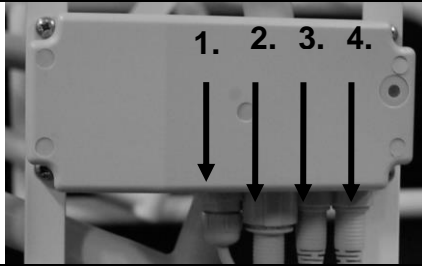

1. Cable for battery.
2. Cable for locking mechanism.
3. Cable for back actuator.
4. Cable for seat actuator.

HEIGHT ADJUSTMENT:

RELEASING THE ARMREST:


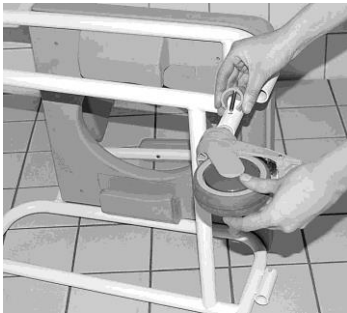


1. Lock all four wheels (by pressing the grey tap on the wheels)
2. Lay down the chair.
3. Take the clip out from the wheel you are adjusting.
4. Grab the wheel and move it until you have fit it in to the right height. All 4 wheels should be placed in the same height.
5. The clip is reinserted (nb! Make sure that the clip is properly pressed down) the procedure is repeated with the other wheels (the chair is turned around to adjust the two "lower" wheels)

Pull the snap release and then you can tilt the armrest.

BACK ADJUSTMENT:

BACK ADJUSTMENT:

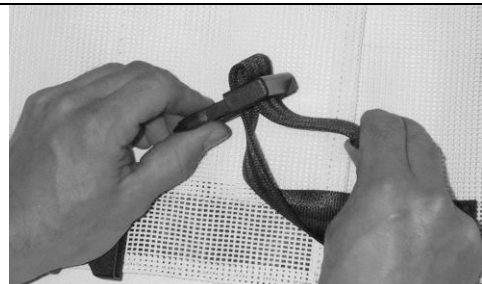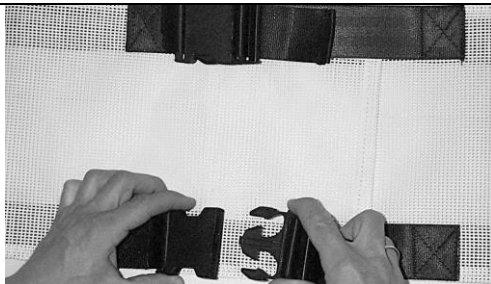


You open the back by pressing the two plastic taps with your left hand, and then with your right hand you pull the buckle out.

The depth of the back is adjusted by pulling the webbing and then tightening it.

Care: Washable at maximum 80 degrees.

USE OF FOOT RESTS

CALF SUPPORT ADJUSTMENT:

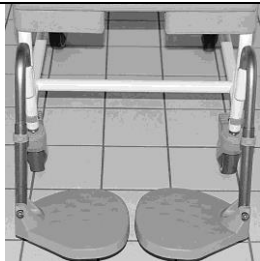

MOUNTING OF FOOT RESTS ON THE CHAIR
 The height of the foot rests are adjusted by taking the clips out and subsequently placing the foot rests in a suitable height and then reinserting the clip.
 Placing the foot rests on one side: You lift the footrests slightly and swing them to either side you want. If you want the foot rests to occupy lesser space you are able to tilt them.


The calf support is fastened with velcro tape on the backside. The depth and the Calf support is adjusted by fastening the velcro tape further in or further out.

Car: Washable at maximum 80 degrees.

USERS GUIDE: M2- FLEXI -TIP SHOWER/COMMODE CHAIR. (4/5)

Made for using when bathing/showering and going to the toilet.

ADJUSTING THE HEADREST.
(ARTICLE NO. 310221):

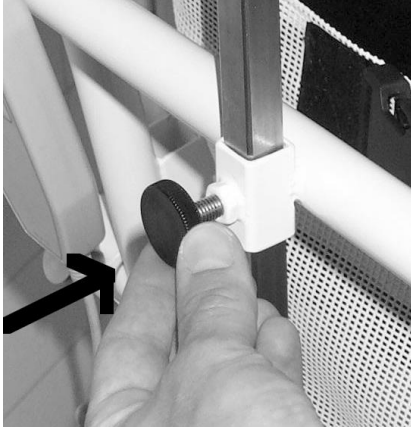


The height of the headrest is adjusted by loosening the fingerscrew and tightening it again when the height is right.

ADJUSTING HEIGHT- AND DEPTH ADJUSTABLE HEADREST
(ARTICLE NO. 310222):




The height and depth of the headrest is adjusted by loosening the two fingerscrews and then tightening them again once they are in the right position.

SIDE SUPPORT ADJUSTMENT:

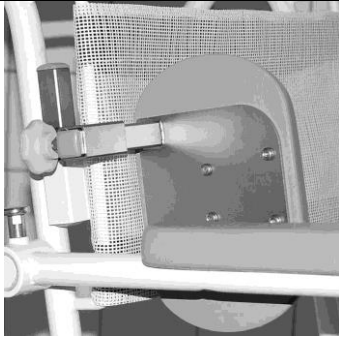

The side supporters are clicked into place in the square pipes on the side of the chair. The depth is adjusted by loosening the fingerscrew and then tightening it again once the height is right. You can also fasten the side supporters so that the cushions are turned horizontally if there is a need for that.

USE OF CHAIR IN NEGATIVE

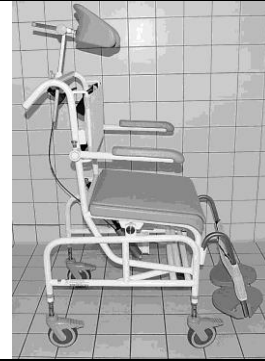

Use of chair in negative position can cause that the user will fall forwards.



WHEN SLIDING THE BUCKET OF THE CHAIR, IT IS IMPORTANT TO SLIDE IT STRAIGHT BACKWARDS, ALONG THE SEAT. OTHERWISE IT CAN BE VERY DIFFICULT TO GET THE BUCKED OF THE CHAIR.

Additional accessories such as buckets etc. See pamphlets for this


WARNING



DO NOT SIT ON THE OUTER 10 CM OF THE SEAT THAT THE CHAIR CAN TIP FORWARD.



DO NOT STAND ON THE FOOTRESTS

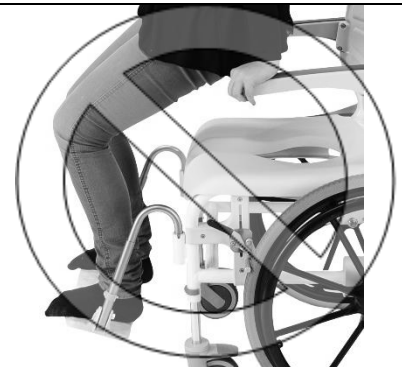

Continued on page 5

USERS GUIDE: M2- FLEXI -TIP SHOWER/COMMODE CHAIR. (5/5)

Made for using when bathing/showering and going to the toilet.

USE OF LOCK



The locking mechanism gives the possibility of locking either one of the actuators. The left one locks the actuator that moves the back rest, and the right one locks the actuator that tilts the seat. If both are locked at the same time none of the actuators will move.

Check with proper intervals the fastening of the wheels to the legs.

Article. No. 310216/310217

Max. User load 125 kg

NB! The clips have to be put all the way through to the bottom of the holes when height adjusting

2 year's complaint right against manufacturing errors.
1 year on batteries.

Terms of claim for frame.

HMN grants 5 years claim rights against frame break-down for all M2-Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that normal wear is not included in the warranty.

Cleaning:

The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes



This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.

09.10.15

Spare parts

Art. No.	Description	Art. No.	Description
310340	Seat, grey		Electric parts
310342	With extra large Seat hole, grey	800310	Remove control (with wire and plug)
310341	Seat lid (std., grey)	800268	Actuator ("engine") for tilt of seat
310343	Seat lid (Intimate, grey)	800311	Actuator ("engine") for tilt of back rest
800181	PU foam on armrest. One unit. Grey	800312	Locking box
310065	Clips for height adjustment, grey	800315	Control box (with plug for battery box)
310165	Clips for height adjustment, magenta	800270	Battery box w/battery (with plug for control box)
800314	Back in net weave for Flexi-tip	800271	Bolt for fastening of actuator (below bottom)
310094	125 mm wheel with 1 leg and 1 clips, 1 pcs.	800272	Bolt with nut for fastening of actuator (on top)
800153	125 mm directional wheel with 1 leg and 1 clips, 1 pcs.	800278	Universal transformer
		800279	English plug
		800280	Euro plug

IN OTHER RESPECTS SHOULD YOU HAVE ANY QUESTIONS THEN YOU ARE MORE THAN WELCOME TO CONTACT US

BEST REGARDS

HMN a/s

Midtager 22, 2605 Brøndby, Denmark

Tel.: +45 43 96 24 00 Fax.: +45 43 96 26 92 E-Mail info@hmn.dk

www.hmn.dk

PRODUCED IN THE EU