


User guide for Shower/Commode chair M2 Gas-Tip. Intended for use a bathing, showering and toilet visit. (1/4)

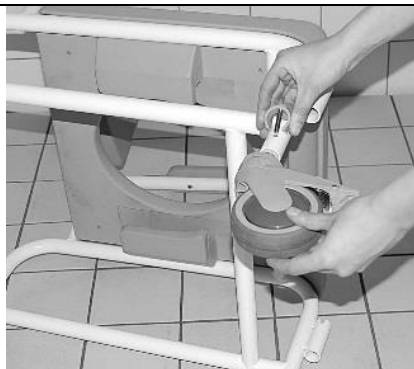

Assembly of wheels:

Lay down the chair. Take the clips out of the leg and insert the wheels into the frame. Make sure that all 4 wheels are in the same height position before inserting the clips. Turn the chair over for easier assembly of the wheels.

Mounting the back:

Attach the back into the frame.




Assembly the back:


Insert the 4 screws and tighten with an Allen wrench. Make sure that all 4 screws are securely tightened. Remember to retighten the screws after a month. The screw shall be discarded after the mounting and you shall use the screws to the next mounting. (Art. No 800290)

Mounting/Disassembly of armrest

Pull the snap release and then you can tilt the armrest.

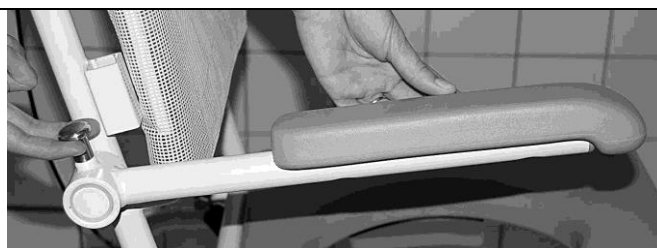

Locking the wheel

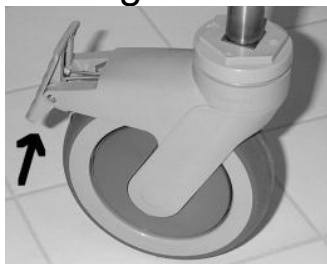

Before you use the chair for bathing/showering, you must make sure you apply the brake on all four wheels.






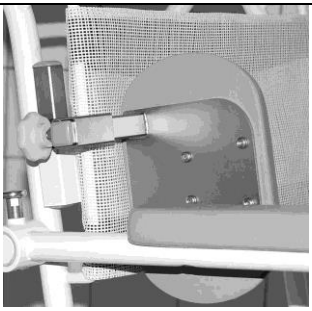
Directional controlled wheel



The right front wheel can be locked the direction. Use of this makes it easier to start and moving the chair.


Continued on page 2

**User guide for Shower/Commode chair M2 Gas-Tip.
Intended for use a bathing, showering and toilet visit.
(2/4)**

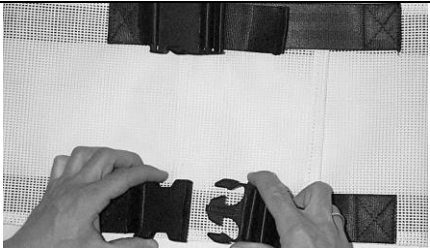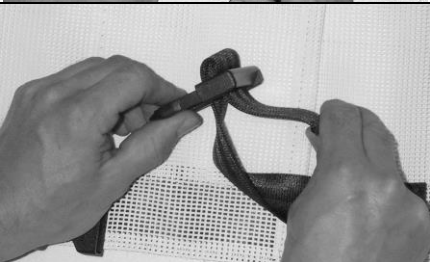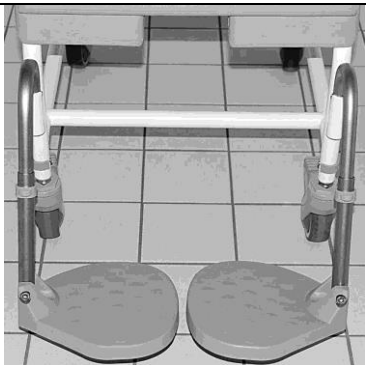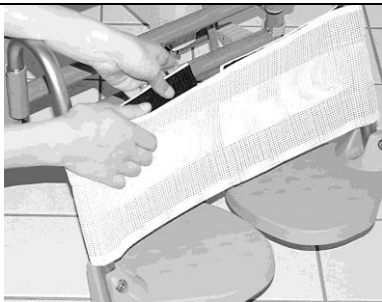Back adjustment	
	<p>You open the back by pressing the two plastic taps with your left hand, and then with your right hand you pull the buckle out.</p> <p>The depth of the back is adjusted by pulling the webbing and then tighten it.</p> <p>Care: Washable at maximum 80 degrees.</p>
	
<p align="center">Mounting the footrests</p> <p>The height of the footrests are adjusted by taking the clips out and subsequently placing the footrests in a suitable height and then reinserting the clip. Placing the footrests on one side: You lift the footrests slightly and swing them to either side you want. If you want the footrests to occupy lesser space you are able to tilt them.</p>	
	<p align="center">Calf support adjustment</p> <p>The calf support is fastened with velcro tape on the backside.</p> <p>The depth and the Calf support is adjusted by fastening the velcro tape further in or further out.</p> <p>Care: Washable at maximum 80 degrees.</p>
<p align="center">The pan/bucket</p> <p>When sliding the bucket of the chair, it is important to slide it straight backwards, along the seat. Otherwise it can be very difficult to get the bucket of the chair.</p>	
	<p align="center">Side support adjustment</p> <p>The side supporters are clicked into place in the square pipes on the side of the chair.</p> <p>The depth is adjusted by loosening the finger screw and then tightening it again once the height is right. You can also fasten the side supporters so that the cushions are turned horizontally if there is a need for that.</p>
Continued on page 3	

User guide for Shower/Commode chair M2 Gas-Tip. Intended for use a bathing, showering and toilet visit. (3/4)

Adjusting the headrest (Art. No. 310221)

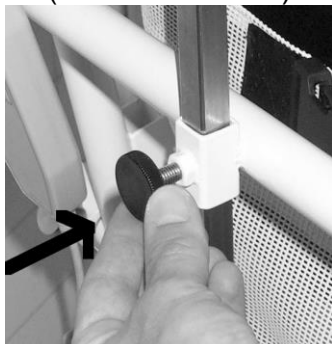

The height of the headrest is adjusted by loosening the finger screw and tightening it again when the height is right.

Adjusting H+D headrest (Art. No. 310222)




The height and depth of the headrest is adjusted by loosening the two finger screws and then tightening them again once they are in the right position.

Use of chair in negative

Use of chair in negative position can cause that the user will fall forwards. Use the mounting against the negative tip, the risk can be eliminated (art. 800251)

The negative tip is **only** for the M2 Gas-Tip chair!

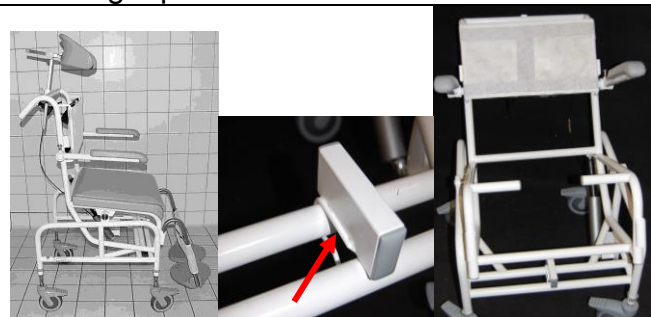


WARNING



Do not sit on the outer 10 cm of the seat that the chair can tip forward.



Do not stand on the footrests.

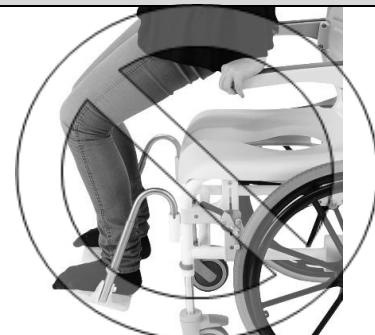

Art. no

Standard seat:

310201/310201-B, 310621/310621-B

Max. Load 140 kg

Attention! Push the Clips to the bottom when adjusting heights.

2 Years complaint rights against manufacturing errors.

Terms of claim for frame.

HMN grants 5 years claim rights against frame break-down for all M2 Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that the normal

Seat with extra large hole:

310202/310202-B, 310622/310622-B

Cleaning:

The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.

This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.



See the spare parts on page 4

User guide for Shower/Commode chair M2 Gas-Tip.
Intended for use a bathing, showering and toilet visit.
(4/4)

Art. No	Spare Parts	Art. No	Spare Parts
310282	Back-Rest (Grey net-weave) from jan. 2011	310076	White nylon bush for armrest
310252	Back-Rest (white net-weave) to jan 2011	310254	Quick release lock for armrests
800326	Height, width adjustment bar	800294	Bolt and mounting for fastening of frame
800327	Screws for mounting of adjustment bar	310276	Foot rest, right
800546	Screws for mounting of adjustment bar for height, depth and width	310277	Foot rest, left
800191	Grey Headrest cushion	800170	Clips, long
800328	Mounting for the depth bar	800169	Bolt for foot plate with nut
800262	Vertical tube for headrest, height and depth adjustable	310226	Leg support (White nylon with velcro)
800027	Finger screw, grey	800233	Upper frame
800249	Grey ferule for headrest	800156	Bottom frame
310094	125 mm wheel with leg on. 1 pcs.	800244	Ferule for round armrest, grey
800277	100 mm wheel with leg on. 1 pcs.	800012	Screws for fastening of armrest
310065	Clip for height adjustment, grey	800082	Bolt for armrest
800013	Bush, wheel	Gas spring parts	
310340	Standardseat, grey	310265	Gas spring
310341	Seat lid for std. seat, grey	310266	Brake lever
310342	Seat with extra large hole, grey	310267	Brake cable
310343	Seat lid for seat with extra large hole, grey	310268	Lockingmchanism (for fitting of brake cable on gas spring)
310352	Armrest for M2 with quick release & grey pu foam, right		
310353	Armrest for M2 with quick release & grey pu foam, left	310262	Bolt with nut for fastening of gas spring (on top)
800181	Pu foam	310263	Bolt for fastening of gas spring (below bottom)

HMN A/S

Midtager 22, 2605 Brøndby, Denmark
Tel. +45 43 96 24 00, Fax. +45 43 96 26 92
E-mail. info@hmn.dk Website. www.hmn.dk




DK: Brugsanvisning
 S: Bruksanvisningen
 N: Bruker veiledning
 EN: User guide
 D: Gebrauchsanweisung
 FIN: Käyttöohje
 E: Instrucciones de uso
 F: Manuel
 I: Manuale
 NL: Gebruiksaanwijzing

- Beslag mod negativ tip
- Beslag mod negativ spets
- Beslag mod negativ tips
- Bracket for avoid negative tip
- Klammer gegen negative Neigung
- Beslag mod negatiivinen kärki
- Beslag inflexión negativa mod
- Beslag mod pointe négative
- Beslag punta negativo mod
- Beslag mod negatieve tip

Art: 800251



- Montering af beslag
- Montering av beslag
- Montering av beslag
- Mounting brackets
- Montage der Klammer
- Kiinnityskannattimet
- Los soportes de montaje
- Supports de montage
- Staffe di montaggio
- Montagebeugels

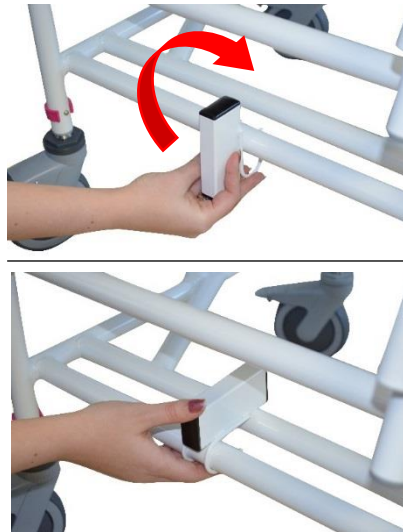

- Sættes foran på stolen
- Put framför stolen
- Satt foran stolen
- Put in front of the chair
- Montage vorne am Rahmen
- Laita edessä tuolin
- Colocar delante de la silla
- Mettre en avant de la chaise
- Mettere di fronte alla sedia
- In de voorkant van de stoel

- Må **kun** anvendes på M2 Gas-tip
- Använd **endast** M2 Gas-tip
- Bruk **kun** M2 Gas-tip
- Is **only** for the M2 Gas-Tip chair
- Kann **nur** auf dem M2 Gas-tip angewendet werden
- Käytä **vain** M2 Gas-tip
- Utilice **sólo** M2 Gas-tip
- Utilisez **uniquement** M2 Gas-tip
- Utilizzare **solo** M2 Gas-tip
- Gebruik **alleen** M2 Gas-tip

