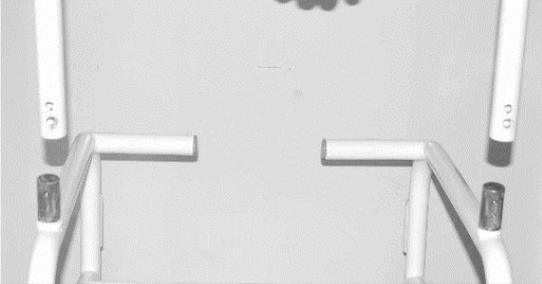
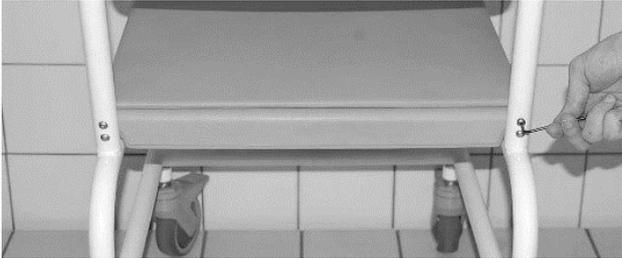
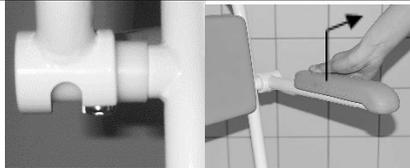


User guide for Shower/Commode chair M2 Mini with 20”

Intended for use a bathing, showering and toilet visit.
(1/5)

<p>Mounting the back:</p> <p>Attach the back into the frame.</p>	
	<p>Assembly the back:</p> <p>Insert the 4 screws and tighten with an Allen wrench. Make sure that <u>all</u> 4 screws are securely tightened. Remember to retighten the screws after a month. The screw shall be discarded after the mounting and you shall use the screws to the next mounting. (Art. No 800290)</p>
<p>Mounting/Disassembly of armrest</p> <p>For mounting/disassembly of armrests; pull up the armrest and out to the side</p>	
	<p>Adjustment of armrests</p> <p>The height of the armrest are adjusted by loosening the umbraco screw and then moving the armrest to the wished height and then tightening the screw again.</p>
<p>Mounting the back wheels</p> <p>Lay down the chair. The bolt is unscrewed and the wheel taken of. The bushings to the 20-inch wheels are taken off with a wrench key and moved to the desired height.</p>	
<p>Continued to page 2</p>	

User guide for Shower/Commode chair M2 Mini with 20”

Intended for use a bathing, showering and toilet visit.

(2/5)



Assembly of wheels:

Lay down the chair. Take the clips out of the leg and insert the wheels into the frame. Make sure that the 2 wheels are in the same height position before inserting the clips.

The front wheels can be put in a position, which is one (1) position different from the 24 inch wheels.

(If you want the chair to be slightly tilted)



Mounting of anti-tip bar

The bushings to the 24-inch wheels are taken off with a wrench key and moved to the desired height.

The anti-tip bar is moved to the same position as the 24-inch wheels.

The clips holding the front wheels are taken out, and the front wheels are moved to the desired height.

Double check that all nuts and bolts are tightened and that all the clips are pressed in place.

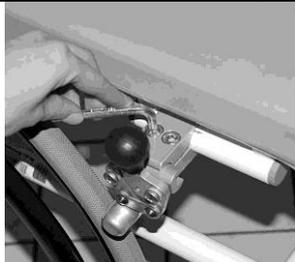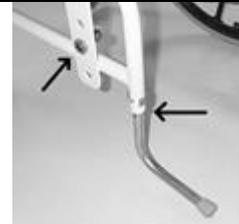

Mounting/Adjusting the handbrake


Put the hook (The big one) on the chair, put the brake on the other side and tight the two umbraco keys.

With the help of an umbraco key you tighten the screws so they give the wanted brake impact. Tighten the bolts well!

Locking the wheels:

Before use of the chair bathing, showering or lifting persons into the chair, make sure that all 4 wheels are locked.

To lock the chair, press down the grey plate as shown on the picture to the left.

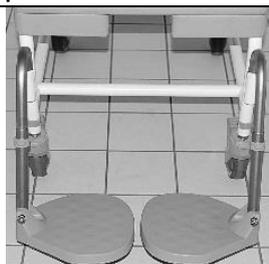




Mounting the footrests:

Adjust the height of the footrests by taking the clips out, placing the footrests in the wished position and putting the clips back in.


Footrests are to be placed on the front of the frame in the footrests holders.

Continued on page 3

User guide for Shower/Commode chair M2 Mini with 20"
 Intended for use a bathing, showering and toilet visit.
 (3/5)

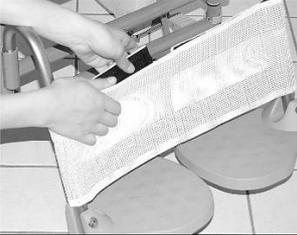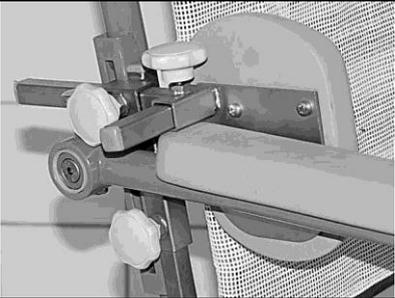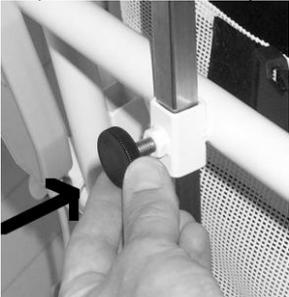	<h3>Adjusting of push handles</h3> <p>Take out the clips and move the handle to the wished position. Put the clips back in. NB! There are holes so the handles can be angled backwards or to the sides so the chair is easier to maneuver.</p>
	
<h3>Mounting of calf support</h3> <p>The calf support is fastened with Velcro tape on the backside. The depth and the calf support is adjusted by fastening the Velcro tape further in or further out. Care: Washable at max. 80 degrees</p>	
	<h3>Mounting of side supports</h3> <p>The side supports are put in the square tube on the chair frame. Height/Depth/Width is adjusted by loosening the finger screw and then holding the side support in the wished position and then tightening the finger screw again. If wished you can place the foam pillow horizontal</p>
<h3>Adjusting the headrest</h3> <p>(Art. No. 310221)</p>  <p>The height of the headrest is adjusted by loosening the finger screw and tightening it again when the height is right.</p>	<h3>Adjusting H+D headrest</h3> <p>(Art. No. 310222)</p>  <p>The height and depth of the headrest is adjusted by loosening the two finger screws and then tightening them again once they are in the right position.</p>
<p>Continued on page 4</p>	

Intended for use a bathing, showering and toilet visit.
(4/5)
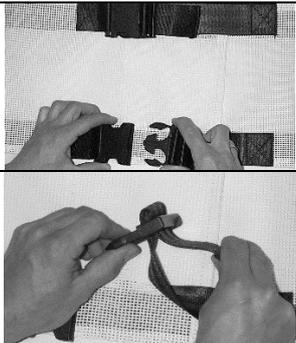
<p>Mounting of hipstrap</p> <p>The two pars are mounted on the back rest tubes. The hipstrap is closed and opened with Velcro.</p>		
	<p>Adjusting the back rest</p> <p>You open the back by pressing the two plastic taps with your left hand, and then with your right hand you pull the buckle out. The depth of the back is adjusted by pulling the webbing and then tighten it. Care: Washable at maximum 80 degrees.</p>	
<p>WARNING </p>		
<p style="text-align: center;">Risk of rolling or glide-out! To avoid the chair can rolls or glider in transfer, to all 4 wheels be locked! ATTENTION! If there is a smooth surface, there is still a risk of the chair rolling or gliding,</p>		
	<p>Do not sit on the outer 10 cm of the seat that the chair can tip forward.</p> <p style="text-align: center;">←</p>	
<p style="text-align: center;">Art. No:</p>	<p style="text-align: center;">310501/310501-B</p>	
<p style="text-align: center;">Max. Load 80 kg</p> <p>Attention! Push the Clips to the bottom when adjusting heights.</p> <p>2 Years complaint rights against manufacturing errors.</p> <p style="text-align: center;">Terms of claim for frame.</p> <p>HMN grants 5 years claim rights against frame break-down for all M2 Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that the normal</p>	<p style="text-align: center;">Cleaning:</p> <p>The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.</p> <p>This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.</p> <div style="text-align: right;">  </div>	
<p>See the spare parts on page 5</p>		

Intended for use a bathing, showering and toilet visit.
(5/5)

Art. No	Spare Parts	Art. No	Spare Parts
800179	Mini seat (light grey)	800183	Armrest set with pu foam pillows, set w/ 2 pieces
800180	Seat cover	800184	100 mm wheel with leg, 1 pcs.
310065	Clips for height adjustment, grey	800185	100 mm wheel with leg. Set with 4 pcs.
310165	Clips for height adjustment, magenta	310081	Finger screw for head support holder
800181	Pu foam pillow for armrests, 1 pcs (light grey).	800186	Push handle, 1 pcs
800012	Screw for fastening of armrest pillows.	800187	Umbra co screw for height adjustment for armrest, 1pcs
310076	Nylon bush for armrests	800188	20" wheel for right side
800082	Bolt for armrest	800189	20" wheel for left side
800013	Nylon bush for leg	800191	Pu foam pillow for head rest (light grey)
800182	White net weave back rest for Mini	800192	Pu foam pillow for side support (light grey)
<p>HMN A/S Midtager 22, 2605 Brøndby, Denmark Tel. +45 43 96 24 00, Fax. +45 43 96 26 92 E-mail. info@hmn.dk Website. www.hmn.dk</p>			

