

IMPORTANT!
**WHEN CHARGING, DO NOT TURN ON THE SWITCH
BEFORE THE CHARGER IS CONNECTED TO THE
CHAIR.**

**THE BATTERY SHOULD BE CHARGED 24 HOURS
THE FIRST TIME.**

**WHEN THE BATTERY HAS RUN DOWN THERE IS A
BEEP SOUND FROM THE CONTROL BOX WHEN THE
ELECTRICAL PARTS ARE IN USE.**

**WHEN THE BEEP SOUND COMES, YOU HAVE TO
RECHARGE THE BATTERY AS IT CAN BE DAMAGED
IF YOU LET IT RUN 100 % OUT OF POWER.**








**(IF THE BATTERY HAS RUN OUT OF POWER, YOU
CANNOT USE THE ENGINE BY PUTTING THE
CHARGER IN THE WALL PLUG AND THE CHAIR.
A FULL RECHARGE TAKES TYPICALLY 8 HOURS.)**

**IF THE LIGHTBOLT ON THE CHARGER LIGHTS
GREEN WHEN CONNECTED, IT IS DUE TO A
MALFUNCTION OF THE BATTERY, AND THEREFORE
IT NEEDS TO BE CHANGED.**

**DO NOT CHARGE THE CHAIR WHILE BEING USED IN
AREAS WITH WATER.**

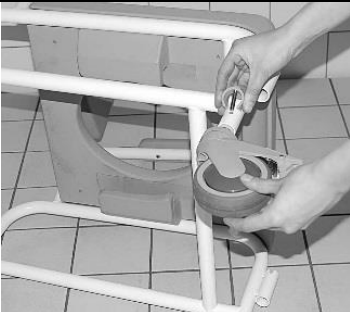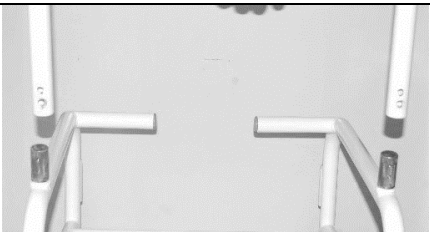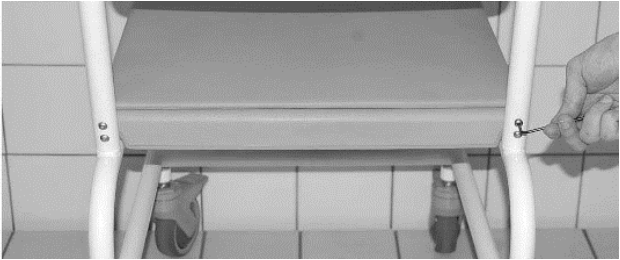


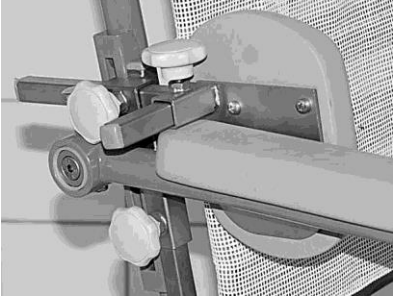
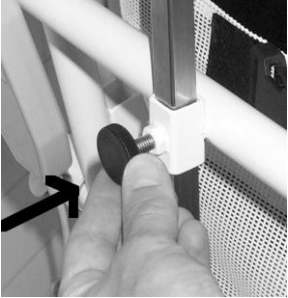


**DO NOT USE THE ELECTRICAL FUNCTIONS ON
THE CHAIR WHILE SPRAYING WITH WATER.**

**User guide for Shower/Commode chair M2 Mini El-Tip.
Intended for use a bathing, showering and toilet visit.
(2/5)**







	<p align="center">Assembly of wheels:</p> <p>Lay down the chair. Take the clips out of the leg and insert the wheels into the frame. Make sure that <u>all</u> 4 wheels are in the same height position before inserting the clips. Turn the chair over for easier assembly of the wheels.</p>
<p align="center">Mounting the back:</p> <p align="center">Attach the back into the frame.</p>	
	<p align="center">Assembly the back:</p> <p>Insert the 4 screws and tighten with an Allen wrench. Make sure that <u>all</u> 4 screws are securely tightened. Remember to retighten the screws after a month. The screw shall be discarded after the mounting and you shall use the screws to the next mounting. (Art. No 800290)</p>
<p align="center">Mounting/Disassembly of armrest</p> <p>Pull the snap release and then you can tilt the armrest.</p>	
	<p align="center">Adjustment of armrests</p> <p>The height of the armrest are adjusted by loosening the umbraco screw and then moving the armrest to the wished height and then tightening the screw again.</p>
<p align="center">Locking the wheels:</p> <p>Before use of the chair bathing, showering or lifting persons into the chair, make sure that <u>all</u> 4 wheels are locked. To lock the chair, press down the grey plate as shown on the picture to the left.</p>	
	<p align="center">Mounting the footrests:</p> <p>Adjust the height of the footrests by taking the clips out, placing the footrests in the wished position and putting the clips back in. Footrests are to be placed on the front of the frame in the footrests holders.</p>

Continued on page 3


User guide for Shower/Commode chair M2 Mini El-Tip.
 Intended for use a bathing, showering and toilet visit.
 (3/5)

	<h3>Adjusting of push handles</h3> <p>Take out the clips and move the handle to the wished position. Put the clips back in. NB! There are holes so the handles can be angled backwards or to the sides so the chair is easier to maneuver.</p>
	
<h3>Mounting of calf support</h3> <p>The calf support is fastened with Velcro tape on the backside. The depth and the calf support is adjusted by fastening the Velcro tape further in or further out. Care: Washable at max. 80 degrees</p>	
	<h3>Mounting of side supports</h3> <p>The side supports are put in the square tube on the chair frame. Height/Depth/Width is adjusted by loosening the finger screw and then holding the side support in the wished position and then tightening the finger screw again. If wished you can place the foam pillow horizontal</p>
<h3>Adjusting the headrest</h3> <p>(Art. No. 310221)</p>  <p>The height of the headrest is adjusted by loosening the finger screw and tightening it again when the height is right.</p>	<h3>Adjusting H+D headrest</h3> <p>(Art. No. 310222)</p>  <p>The height and depth of the headrest is adjusted by loosening the two finger screws and then tightening them again once they are in the right position.</p>
<h3>Mounting of hipstrap</h3> <p>The two parts are mounted on the back rest tubes. The hipstrap is closed and opened with Velcro.</p>	
<p>Continued on page 4</p>	

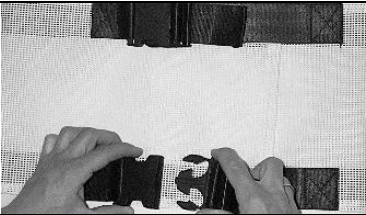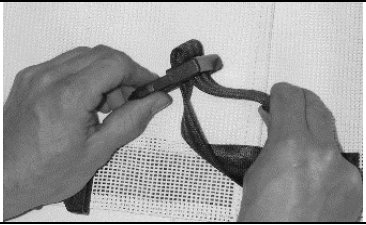

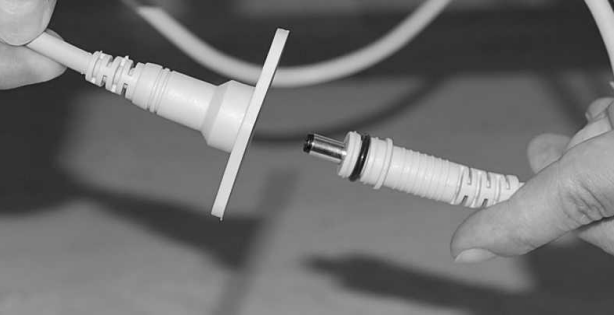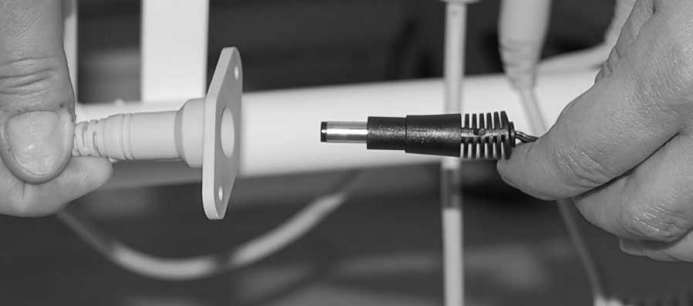User guide for Shower/Commode chair M2 Mini El-Tip.
 Intended for use a bathing, showering and toilet visit.
 (4/5)


	<h3>Adjusting the back rest</h3> <p>You open the back by pressing the two plastic taps with your left hand, and then with your right hand you pull the buckle out. The depth of the back is adjusted by pulling the webbing and then tighten it. Care: Washable at maximum 80 degrees.</p>
	<h3>Tilting</h3> <p>For going up or down the chair is tilted by using the remote-control. If the user is not able to hold on to the armrest by himself, the helper must lay the users hands on his lap before tilting the chair</p> 
	<h3>Charging the battery</h3> <p>Depending on region plug type 1, 2 or 3 slides into the charger. Afterwards you put the transformer in the socket and turn on the power. A full charge takes approximately 8 Hours. When the batteri is running out of power you will hear a "beep" sound. When the battery is being charged, the light bolt on the charger lights orange. When the battery has been fully charged, the light bolt on the battery will light green. Note: If the light bolt on the charger lights green, when it is being connected, it is due to a malfunction, and it needs to be changed.</p>
<h3>Charging the battery</h3>  <p>With the one hand, you grab around the charging plug (Socket) on the chair and with the other hand you pull out the connecting cable for the battery.</p>	<h3>Charging the battery</h3>  <p>With the one hand, you grab around the charging plug (Socket) on the chair and with the other hand, you push the plug from the transformer in place.</p>
<p style="text-align: center;">Continued on page 5</p>	

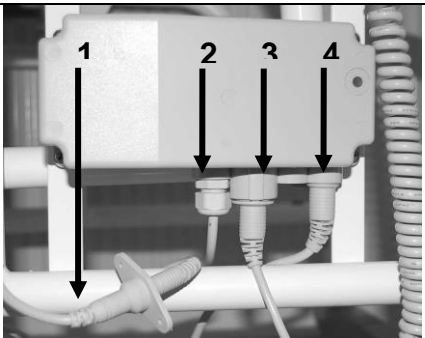User guide for Shower/Commode chair M2 Mini El-Tip. Intended for use a bathing, showering and toilet visit. (5/5)

	<h3 style="text-align: center;">Cables</h3> <ol style="list-style-type: none"> 1. Plug for battery 2. Cable for battery 3. Cable for remote control 4. Cable for actuator ("engine")
---	--

WARNING

	<p style="text-align: center;">Do not sit on the outer 10 cm of the seat that the chair can tip forward.</p> <p style="text-align: center;">←</p>	
<p style="text-align: center;">Do not stand on the footrests</p> <p style="text-align: center;">→</p>		

<p style="text-align: center;">Art. No:</p> <p style="text-align: center;">Max. Load 80 kg</p> <p>Attention! Push the Clips to the bottom when adjusting heights. 2 Years complaint rights against manufacturing errors.</p> <p style="text-align: center;">Terms of claim for frame.</p> <p>HMN grants 5 years claim rights against frame break-down for all M2 Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that the normal</p>	<p style="text-align: center;">310502/310502-B</p> <p style="text-align: center;">Cleaning:</p> <p>The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.</p> <p>This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.</p> 
---	---

Art. No	Spare Parts	Art. No	Spare Parts
800179	Mini seat (light grey)	310081	Finger screw for head support holder
800180	Seat cover	800186	Push handle, 1 pcs
310065	Clips for height adjustment, grey	800187	Umbra co screw for height adjustment for armrest, 1pcs
310165	Clips for height adjustment, magenta	800191	Pu foam pillow for head rest (light grey)
800181	Pu foam pillow for armrests, 1 pcs (light grey).	800192	Pu foam pillow for side support (light grey)
800012	Screw for fastening of armrest pillows.	800193	Finger screw for side support (round)
310076	Nylon bush for armrests	800267	Remote control (including wire and plug)
800082	Bolt for armrest	800269	Control box
800013	Nylon bush for leg	800270	Battery box incl. battery
800182	White net weave back rest for Mini	800268	Actuator ("engine")
800183	Armrest set with pu foam pillows, set w/ 2 pieces	800280	Transformer (Euro plug)
800181	PU foam for the armrest	800272	Bolt with nut for fastening of actuator (on top)
800184	100 mm wheel with leg, 1 pcs.	800271	Bolt for fastening of actuator (below bottom)

HMN A/S

Midtager 22, 2605 Brøndby, Denmark
Tel. +45 43 96 24 00, Fax. +45 43 96 26 92
E-mail. info@hmn.dk Website. www.hmn.dk