


USER GUIDE

Neptun Super Soft 300 KG Shower stool with telescopic legs.

Height adjustment:

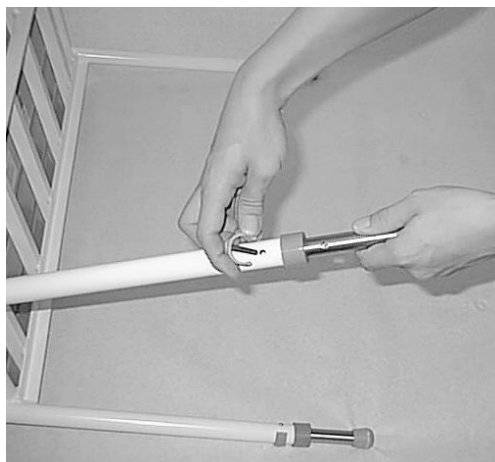

1. Lay the shower stool on the side.
2. Remove the clips while holding the leg.
3. Move the leg to the desired position and attach the clips.
4. Make sure the clips have gone through the hole.
5. Repeat procedure for the remaining three legs.

Mounting of back rest:


Remove the seats.

Place the backrest in the pipes.

Mount the clips on both sides.

The backrest has two positions.

Make sure that the same position is used for both sides.

| Article No. 300300 Maximum user load 300 kg | | Cleaning: The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes | |
|--|---|--|---|
| NB! The clip must be pushed all the way through when adjusting the height 2 year's complaint right against manufacturing errors. | | 22.01.13 |  |
| Spare parts | | | |
| Art. No. | Description | Art. No. | Description |
| 312030 | Neptun Seat | 310065 | Clips for height adjustment, grey |
| 800300 | PU foam back | 310165 | Clips for height adjustment, magenta |
| 800304 | Telescopic leg(w/clip,rubber feet, bush) 1pcs. | | |
| 310017 | Rubber feet for leg | | |
| 800366 | Spacer for rubber feet for shower stool 200/300kg | | |

HMN a/s

Midtager 22, 2605 Brøndby, Denmark

Tlf.: +45 43 96 24 00 Fax. +45 43 96 26 92 E-mail info@hmn.dk

www.hmn.dk

PRODUCED IN THE EEC