

USERS GUIDE

NEPTUN SUPER SOFT HEIGHT ADJUSTABLE SHOWER STOOL

USE OF NEPTUN SUPERSOFT SHOWERSTOOL



STOOL MADE TO SIT ON WHEN WASHING/SHOWERING. THE SHOWER STOOL ALLOWS THE USER TO TRANSFER THEMSELVES USING A SLIDING BOARD

MOUNTING OF SEAT




THE SEAT IS CLICKED ON THE FRAME

THE HEIGHT IS ADJUSTED THE FOLLOWING WAY:

- 1) THE SHOWER STOOL IS PUT ON THE SIDE.
- 2) THE CLIPS IS TAKEN OUT
- 3) THE LEG IS MOVED TO THE DESIRED HEIGHT
- 4) THE CLIPS IS PUT BACK IN THE HOLE (NB! THE CLIPS HAVE TO BE PUT ALL THE WAY TO THE BOTTOM BEFORE USE)
- 5) THE SAME PROCEDURE IS REPEATED ON THE OTHER 3 LEGS.



THE BACK REST IS MOUNTED THE FOLLOWING WAY:

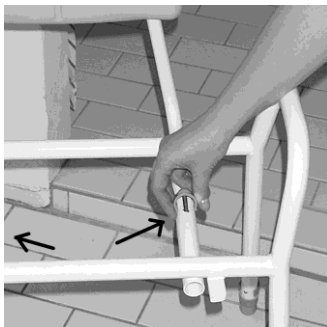

- 1) THE SEAT IS TAKEN OF.
- 2) THE BACK REST IS PUSHED INTO THE 2 BACK PIPES.
- 3) THE BACK REST IS PLACED IN THE DESIRED POSITION(THERE ARE TWO POSITIONS)
- 4) THE CLIPS ARE PUT IN THE HOLES AS SHOWN ON THE PHOTO. ONE CLIP IN EACH OF THE TWO BACK PIPES. (NB! THE CLIPS HAVE TO BE PUT ALL THE WAY TO THE BOTTOM BEFORE USE).
- 5) THE SEAT IS CLICKED BACK ON.

<p style="text-align: center;">Article. No. 312010 Max. user load 150 kg NB! The clips have to be put all the way to the bottom of the holes before use. 2 year's complaint right against manufacturing errors. Terms of claim for frame. HMN grants 5 years claim rights against frame break-down for all M2- Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that normal wear is not included in the warranty.</p>	<p style="text-align: center;">Cleaning: The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes</p> <p style="text-align: center;">22.01.13</p> <div style="text-align: right; font-size: 2em; font-weight: bold;">CE</div>																								
<table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <th style="width: 10%;">Artcl.No:</th> <th colspan="3" style="text-align: center;">SPARE PARTS::</th> </tr> <tr> <td></td> <th colspan="3" style="text-align: center;">NEPTUN SHOWER STOOLS</th> </tr> <tr> <td>310065</td> <td>Clips for height adjustment, grey</td> <td>310017</td> <td>Rubberfeet for telescopic legs</td> </tr> <tr> <td>310165</td> <td>Clips for height adjustment, magenta</td> <td>310021</td> <td>Bush for telescopic legs</td> </tr> <tr> <td>800300</td> <td>Foam Seat for Neptun Super Soft</td> <td>800301</td> <td>PU foam for backrest.</td> </tr> <tr> <td>312033</td> <td>Telescopic legs (1pc. With clips, rubberfeet, bush)</td> <td></td> <td></td> </tr> </table>	Artcl.No:	SPARE PARTS::				NEPTUN SHOWER STOOLS			310065	Clips for height adjustment, grey	310017	Rubberfeet for telescopic legs	310165	Clips for height adjustment, magenta	310021	Bush for telescopic legs	800300	Foam Seat for Neptun Super Soft	800301	PU foam for backrest.	312033	Telescopic legs (1pc. With clips, rubberfeet, bush)			
Artcl.No:	SPARE PARTS::																								
	NEPTUN SHOWER STOOLS																								
310065	Clips for height adjustment, grey	310017	Rubberfeet for telescopic legs																						
310165	Clips for height adjustment, magenta	310021	Bush for telescopic legs																						
800300	Foam Seat for Neptun Super Soft	800301	PU foam for backrest.																						
312033	Telescopic legs (1pc. With clips, rubberfeet, bush)																								

HMN a/s
 Midtager 22, 2605 Brøndby, Denmark
 Tel.: +45 43 96 24 00 Fax. +45 43 96 26 92 E-mail info@hmn.dk
www.hmn.dk
 PRODUCED IN THE EEC