


# USERS GUIDE NEPTUN HEIGHT ADJUSTABLE SHOWER STOOL

Stool made to sit on when washing/showering.


The seat is mounted the following way:

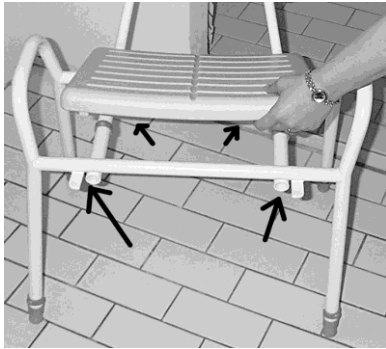


THE SEAT IS CLICKED IN PLACE BETWEEN THE 4 FLAT PIECES OF STEEL (2 PIECES ON EACH SIDE OF THE SHOWER STOOL).

THE HEIGHT IS ADJUSTED THE FOLLOWING WAY:

- 1) THE SHOWER STOOL IS PUT ON THE SIDE.
- 2) THE CLIPS IS TAKEN OUT
- 3) THE LEG IS MOVED TO THE DESIRED HEIGHT
- 4) THE CLIPS IS PUT BACK IN THE HOLE (NB! THE CLIPS HAVE TO BE PUT ALL THE WAY TO THE BOTTOM BEFORE USE)
- 5) THE SAME PROCEDURE IS REPEATED ON THE OTHER 3 LEGS.

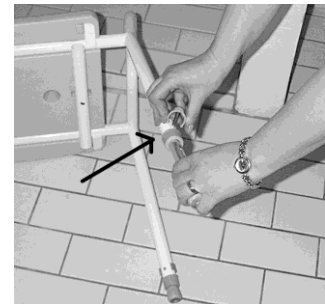

THE BACK REST IS MOUNTED THE FOLLOWING WAY:

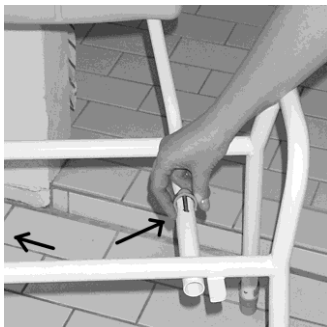

- 1) THE SEAT IS TAKEN OF.
- 2) THE BACK REST IS PUSHED INTO THE 2 BACK PIPES.
- 3) THE BACK REST IS PLACED IN THE DESIRED POSITION(THERE ARE TWO POSITIONS)
- 4) THE CLIPS ARE PUT IN THE HOLES AS SHOWN ON THE PHOTO. ONE CLIP IN EACH OF THE TWO BACK PIPES. (NB! THE CLIPS HAVE TO BE PUT ALL THE WAY TO THE BOTTOM BEFORE USE).
- 5) THE SEAT IS CLICKED BACK ON.

Article. No. 312000/312001

**Max. user load 150 kg**

**NB! The clips have to be put all the way to the bottom of the holes before use.**

2 year's complaint right against manufacturing errors.

**Terms of claim for frame.**

HMN grants 5 years claim rights against frame break-down for all M2-Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that normal wear is not included in the warranty.

Cleaning:

The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes

22.01.13



Artcl.No:	NEPTUN SHOWER STOOLS			SPARE PARTS::	
310065	Clips for height adjustment , grey	310017	Rubberfeet for telescopic legs		
310165	Clips for height adjustment, magenta	310021	Bush for telescopic legs		
312030	Plastic seat for Neptun	312031	Plastic part for backrest.		
312033	Telescopic legs (1pc. With clips, rubberfeet, bush)				

HMN a/s

Midtager 22, 2605 Brøndby, Denmark

Tel.: +45 43 96 24 00 Fax. +45 43 96 26 92 E-mail [info@hmn.dk](mailto:info@hmn.dk)

[www.hmn.dk](http://www.hmn.dk)

PRODUCED IN THE EEC