


USERS GUIDE NEPTUN HEIGHT ADJUSTABLE SMALL SHOWER STOOL

Stool made to sit on when washing/showering.


The seat is mounted the following way:

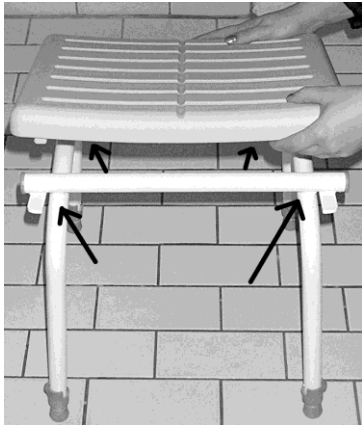

THE SEAT IS CLICKED IN PLACE BETWEEN THE 4 FLAT PIECES OF STEEL (2 PIECES ON EACH SIDE OF THE SHOWER STOOL).

THE HEIGHT IS ADJUSTED THE FOLLOWING WAY:

- 1) THE SHOWER STOOL IS PUT WITH THE BOTTOM UPWARDS.
- 2) THE CLIPS IS TAKEN OUT
- 3) THE LEG IS MOVED TO THE DESIRED HEIGHT
- 4) THE CLIPS IS PUT BACK IN THE HOLE (NB! THE CLIPS HAVE TO BE PUT ALL THE WAY TO THE BOTTOM BEFORE USE)
- 5) THE SAME PROCEDURE IS REPEATED ON THE OTHER 3 LEGS.

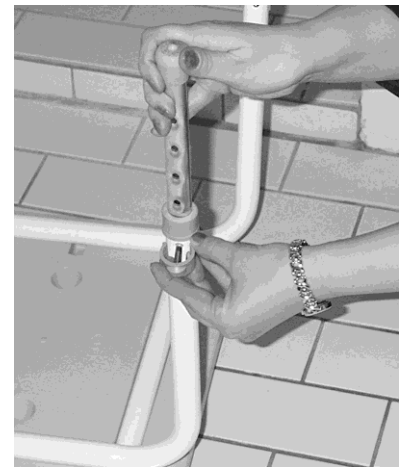

NB! You cannot use a back rest with this item

Article. No. 312100

Max. user load 150 kg

NB! The clips have to be put all the way to the bottom of the holes before use

2 year's complaint right against manufacturing errors.

Terms of claim for frame.

HMN grants 5 years claim rights against frame break-down for all M2-Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that normal wear is not included in the warranty.

Cleaning:

The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes



22.01.13

SPARE PARTS::	
NEPTUN SMALL SHOWER STOOL	
Art.no:	
310065	Clips for height adjustment, grey
310165	Clips for height adjustment, magenta
312030	Plastic seat for Neptun
312130	Telescopic legs (1pc. with clips, rubberfeet, bush)
310017	Rubberfeet for telescopic legs
310021	Bush for telescopic legs
312131	Plastic endings for side of shower stool

HMN a/s

Midtager 22, 2605 Brøndby, Denmark

Tel.: +45 43 96 24 00 Fax. +45 43 96 26 92 E-mail info@hmn.dk

www.hmn.dk

PRODUCED IN THE EEC