

USERS GUIDE: WALL MOUNTED FOLDING SEAT.  
Made for using when bathing/showering.

FOLDING OF SEAT:




To fold the seat, you have to take the cross bar in the front of the seat and lift up.

CLEANING:



To make cleaning easier you can click the seat on and off.

ADJUSTING THE HEIGHT:

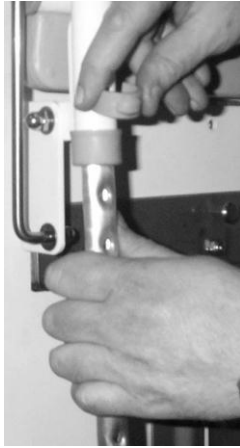

Adjusting the height of the leg is done with clips without the use of tools

RELEASING THE SEAT:




When you use the w  
quick release.

by means of a

ADJUSTING THE HEIGHT OF THE WALL MOUNTING:

MOUNTING THE WALL MOUNTED FOLDING SEAT:

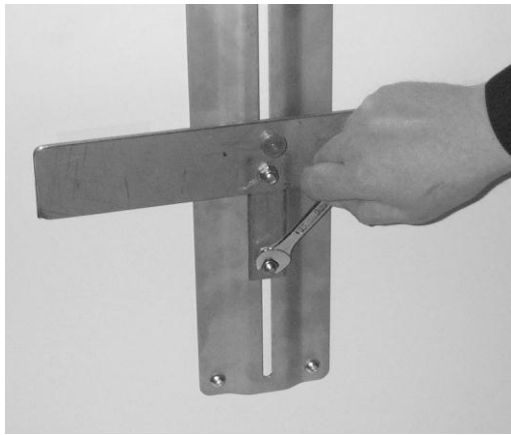

**MOUNTING OF THE WALL MOUNTED FOLDING SEAT IN THE WALL MUST BE DONE BY A PROFESSIONAL (SEE MOUNTING INSTRUCTIONS)**

To adjust the height on the height adjustable wall mounting you have to loosen and tighten the two bolts on the wall mounting (Tighten well)

**Regarding placement of the holes: See enclosed drawing**

Article NO.: 310290, (+ Mounting - Article No.: 310291, 310292)

Max. User load 125 kg

**NB! The Clip must be pushed all the way through when height adjusting.**

**2 year's complaint right against manufacturing errors.**

Cleaning: When disinfecting:

**What can be used; Ethanol and Propanoll dissolution. Otherwise you use ordinary obliging non-alcoholic cleaning material. Maximum 70 degrees**

Article No.:	Spare parts for wall mounted folding seat
312030	Plastic Neptun seat for wall mounted folding seat
800077	Telescope leg (1 unit with clips, ferrule, bush)
310065	Clip for height adjustment, grey
310165	Clips for height adjustment, magenta
310017	Ferrule (for telescope leg)
310021	Bush (for telescope leg)
310075	Plastic ferrule (for top of leg)
312131	Plastic ferrule (for side of wall mounted folding seat)
310254	Quick release for wall mounting
800078	Bolts for height adjusting of wall mounting

23.01.13



**HMN a/s**  
 Midtager 22, 2605 Brøndby, Denmark  
 Tel.: +45 4396 2400 Fax. +45 4 96 2692  
 E-mail [info@hmn.dk](mailto:info@hmn.dk) [www.hmn.dk](http://www.hmn.dk)