


USERS GUIDE HEIGHT ADJUSTABLE SHOWER STOOL

Stool designed for sitting on when bathing/washing/showering

THE HEIGHT IS ADJUSTED THE FOLLOWING WAY:

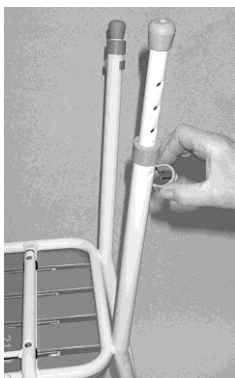

- 1) THE STOOL IS TURNED UPSIDE DOWN.
- 2) THE CLIP IS PULLED OUT.
- 3) THE LEG IS ADJUSTED IN A SUITABLE HEIGHT.
- 4) THE CLIP IS REINSERTED (NB! THE CLIP MUST BE PUSHED ALL THE WAY THROUGH BEFORE USE)
- 5) THE SAME PROCEDURE IS REPEATED ON THE OTHER 3 LEGS.

INCLINING THE SEAT

THE LEGS ARE ADJUSTED AS MENTIONED ABOVE.

WITH THE FRONT CLIP PLACED IN THE LOWER HOLE AND THE REAR CLIP IN THE UPPER HOLE, THE SEAT HEIGHT IS 47 CM AT THE FRONT AND 52 CM AT THE BACK (7° ANGLE)

WITH THE FRONT CLIP PLACED IN THE LOWER HOLE AND THE REAR CLIP IN THE INTERMEDIATE HOLE, THE SEAT HEIGHT IS 47 CM AT THE FRONT AND 50 CM AT THE BACK (3,5° ANGLE)

MOUNTING OF THE BACK



1. Turn the stool upside-down.
2. Attach the two crossbars with the 6 screws. (See picture on the right).
3. Attach the backrest using the finger screws and distance pieces. (See picture on the left)



Article No. 310145

Maximum user load 200 kg

NB! The clip must be pushed all the way through when adjusting the height

2 year's complaint right against manufacturing errors.

Cleaning:

The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes

22.01.13



SPARE PARTS:

Art. No.	Description	Art. No.	Description
310170	Grey slat, Middle	800366	Spacer for rubber feet for shower stool 200/300kg
310171	Grey slat, outer	800219	Complete seat for 200 kg's showerstool
310172	Telescopic leg(w/clip,rubber feet, bush) 1pcs.	310065	Clips for height adjustable, grey
800002	Rivets for slats for 200 kg's shower stool	310165	Clips for height adjustable, magenta

HMN a/s

Midtager 22, 2605 Brøndby, Denmark

Tlf.: +45 43 96 24 00 Fax. +45 43 96 26 92 E-mail info@hmn.dk

www.hmn.dk

PRODUCED IN THE EEC