


USERS GUIDE HEIGHT ADJUSTABLE SHOWER STOOL

Stool designed for sitting on when bathing/washing/showering

THE HEIGHT IS ADJUSTED THE FOLLOWING WAY:

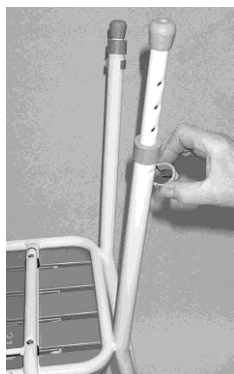

- 1) THE STOOL IS TURNED UPSIDE DOWN.
- 2) THE CLIP IS PULLED OUT
- 3) THE LEG IS ADJUSTET IN A SUITABLE HEIGHT
- 4) THE CLIP IS REINSERTED (NB! THE CLIP MUST BE PUSHED ALL THE WAY THROUGH BEFORE USE)
- 5) THE SAME PROCEDURE IS REPEATED ON THE OTHER 3 LEGS.

MOUNTING OF THE BACK



A BACK SET CONSISTS OF:
1 BACK, 4 SCREWS, 4 SPACERS

- 1) THE STOOL IS TURNED UPSIDE DOWN
- 2) THE SPACERS ARE PLACED BETWEEN THE STOOL AND THE BACK (BELOW THE 4 HOLES)
- 3) THE 4 SCREWS ARE TIGHTENED WELL!



NB! For article 310023 (Stool without handles) the back is not available

Article No. 310023/310120/310122/310123/

Maximum user load 150 kg
NB! The clip must be pushed all the way through when adjusting the height

2 year's complaint right against manufacturing errors.

Cleaning:

The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes

CE

This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.

09.10.15

Article.		SPARE PARTS:	
HEIGHT ADJUSTABLE STOOLS		STOOL WITH HOLE FOR INTIMATE WASHING	
	Slats = Same as stools with fixed legs		Telescopically legs etc are the same to the stool only the slats are different.
310016	Telescopically stool, loose leg with everything	310161	Small outer slat
310017	Telescopically stool, ferrule	310162	Small middle slat
310019	Telescopically stool, clips		BACK
310021	Bush for Telescopically stool	310153	Back slat top
310160	Rivet	310154	Back slat bottom
310065	Clips for height adjustable, grey	310182	Back spacer
310165	Clips for height adjustable, magenta	310184	Back machine screw
		310163	Ferrule ø 19 mm

HMN a/s

Midtager 22, 2605 Brøndby, Denmark

Tlf.: +45 43 96 24 00 Fax. +45 43 96 26 92 E-mail info@hmn.dk

www.hmn.dk

PRODUCED IN THE EEC