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**EN:** When mounting the wheels, the telescopic legs is placed in the pipes and the clips is mounted in suitable and uniform height.

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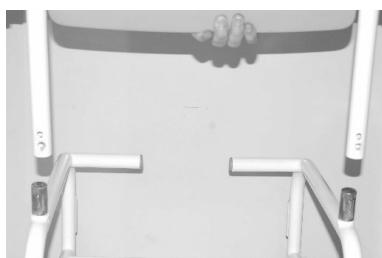
**EN:** Screws are tightened using the Allen wrench when mounting the back. Make sure the screws are tight to ensure stability.

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**EN:** The screws are tightened using an Allen wrench. Make sure the screws are tightened securely to ensure that the spine stability is assured. Remember to tighten your back after about 1 month. The screws should be discarded when dismantling, and will use new screws when next on mounting.

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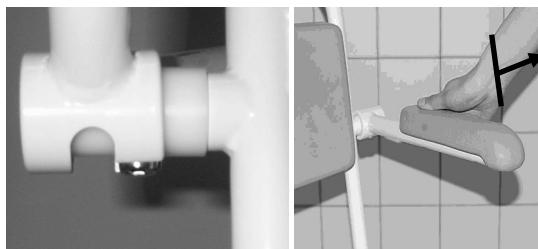
**EN:** For installing / removing the armrests; tag armrests up and out to the side.

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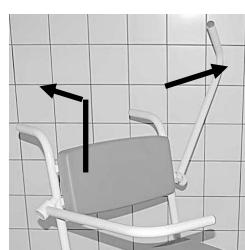
**EN:** For installing / removing the armrests; remove the armrest up and tilted to the side

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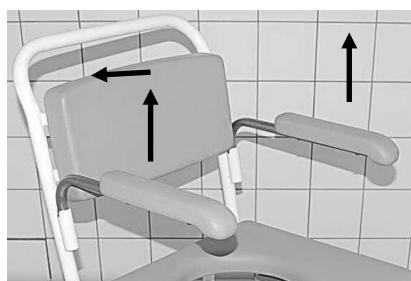
**EN:** For installing / removing the armrests; remove the armrest up. Can also be pulled off to the side.

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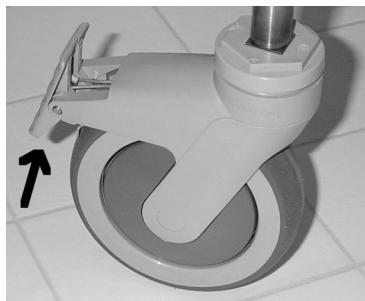
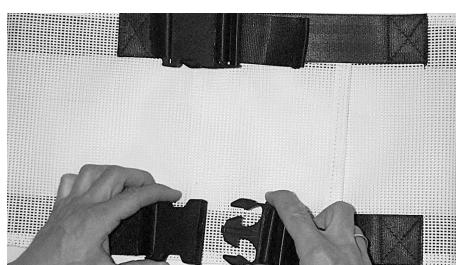
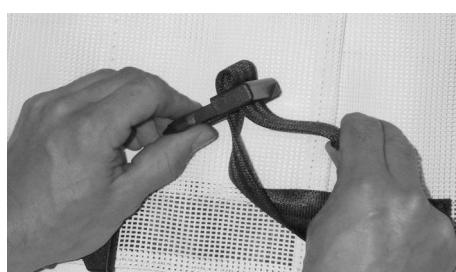
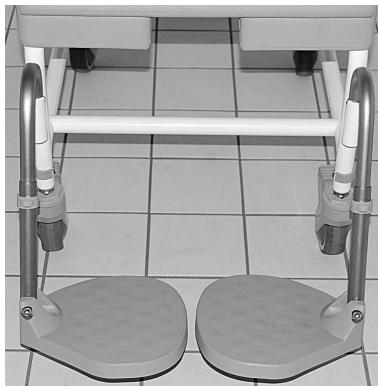
**EN: Art. nr.**

<p><b>Standard seat:</b> Max. Load 150 kg 310720-B, 310721-B, 310722-B</p> <p><b>Attention!</b> Push the Clips to the bottom when adjusting heights.</p> <p>2 Years complaint rights against manufacturing errors. (And rights of complaint for rust)</p> <p><b>Frame warranty</b></p> <p>HMN grants 5 years frame warranty against frame break-down for all M2 Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that the normal wear and tear is not covered under the warranty.</p>	<p><b>Horseshoe seat:</b> Max. Load 150 kg 310720-B, 310721-B, 310722-B</p> <p><b>Cleaning:</b></p> <p>The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.</p> <p><b>Life expectancy: 10-15 years.</b></p> 
---	--

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EN:

This product should be recycled following national regulation. After disassembly of the product, sort the components based on what they are made of and recycle them individually. Please contact your local recycling place for guidance if necessary.

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