NIELSEN LINE

Bade-/Toiletstol
 Dusch- / toalettstol
 Dusj-/ toalettstol
 Shower- / commode chair
 Dusch-/ Toilettenstuhl
 Shower- / lipasto tuoli
 Ducha- / silla con orinal
 Douche- / chaise percée
 Doccia- / sedia comoda
 Douche-/ toiletstoel

DK: Betjeningsvejledning
 S: Bruksanvisningen
 N: Bruker veiledning
 GB: User guide
 D: Bedienungsanleitung
 FIN: Käyttöohje
 E: Instrucciones de uso
 F: Manuel
 I: Manuale
 NL: Gebruiksaanwijzing

DK: Forventet levetid: 10 år.
 S: Förväntad livslängd: 10 år.
 N: Forventet levetid: 10 år.
 GB: Life expectancy: 10 years.
 D: Lebenserwartung: 10 Jahre.
 FIN: Elinaika: 10 vuotta.
 E: Esperanza de vida: 10 años.
 F: Espérance de vie: 10 ans.
 I: Aspettativa di vita: 10 anni.
 NL: Levensverwachting: 10 jaar.

- | | | | |
|--|---|---|---|
| <p>1</p> <ul style="list-style-type: none"> • Produktbeskrivelse • Produktbeskrivning • Produktbeskrivelse • Product Description • Produktbeschreibung • Tuotekuvaus • Descripción del producto • Description du produit • Descrizione del prodotto • Productbeschrijving | <p>2</p> <ul style="list-style-type: none"> • Reservedele • Reservdelar • Reservedeler • Spare parts • Ersatzteile • Varaosat • Refacciones • Pièces de rechange • Pezzi di ricambio • Reserve-onderdelen | <p>3</p> <ul style="list-style-type: none"> • Samlevejledning • Monteringsanvisningar • Monteringsveiledning • Assembling instruction • Montageanleitung • Kokoaminen suuntiin • Direcciones montaje • Instructions d'installation • Istruzioni per l'installazione • Montagehandleiding | <p>4</p> <ul style="list-style-type: none"> • Betjeningsvejledning • Bruksanvisning • Bruker veiledning • User guide • Bedienungsanleitung • Käyttöohje • Instrucciones de uso • Manuel • Manuale • Gebruiksaanwijzing |
| <p>5</p> <ul style="list-style-type: none"> • Ekstraudstyr • Tillval • Valgfrie utstyr • Accessories • Zubehör • Asusteet • Accesorios • Accessoires • Accessori • Toebehoren | <p>6</p> <ul style="list-style-type: none"> • Vedligeholdelse/Garanti • Företrådande/Garanti • Vedlikehold/Garanti • Maintenance/Warranty • Wartung/Garantie • Huolto/Takuu • Mantenimiento/Garantia • Maintenance/Garantie • Manutenzione/Garanzia • Onderhoud/Garantie | <p>7</p> <ul style="list-style-type: none"> • Forebyggende Eftersyn • Rekonditionerings Instruktion • Forebyggende vedlikeholdssjekk • Preventative Maintenance Check • Präventive Wartungsprüfung • Ennaltaehkäisevä huoltotarkastus • Mantenimiento Preventivo Chequeo • La vérification de maintenance préventive • Controllo di Manutenzione Preventiva • Preventieve onderhoudscontrole | |

1

Nielsen Line

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
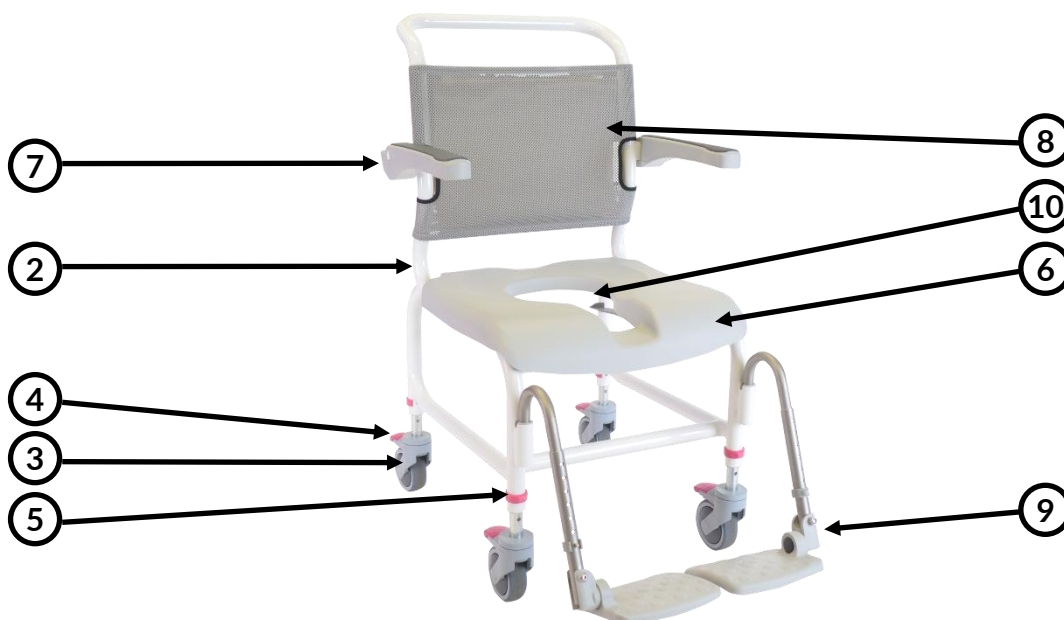
Preventative Maintenance Check:

Applies to the following Art.no: (313031, 313030, 313032, 313033)

To ensure that the aid functions safely for the user, we recommend that the following inspections be carried out.

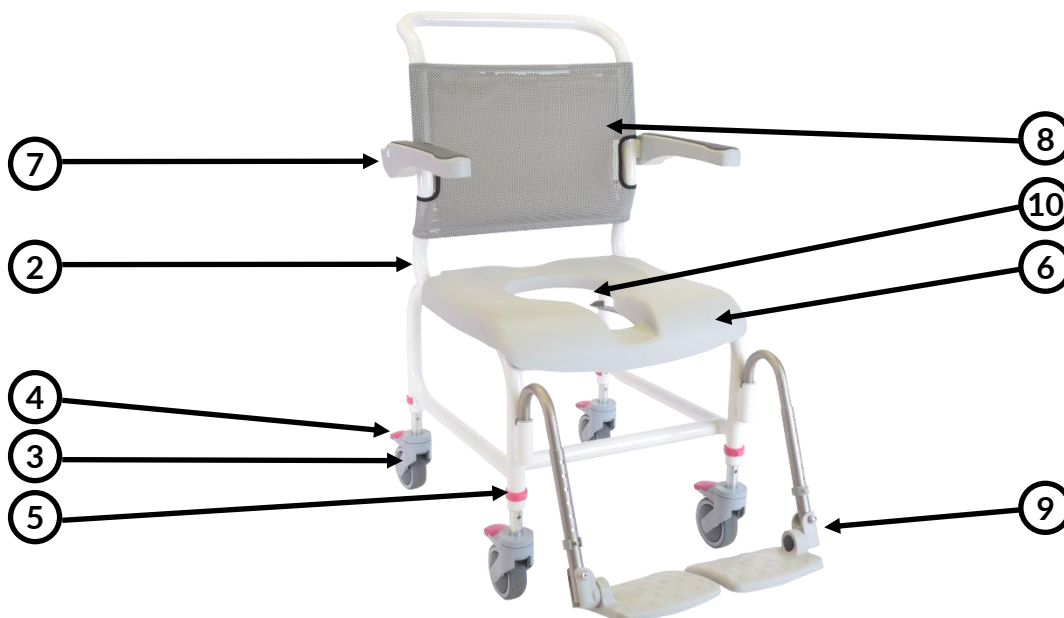1. Check that the frame is without breaks and the chair is intact.
2. Check that the back frame is correctly mounted with 2 screws on each side.
3. Check that the wheels on the chair can roll freely.
4. Ensure that the brakes on the wheels are functioning.
5. Check that the clips for height adjustment of the chair's wheels are not defective.
6. Check that the seat is intact and can be mounted correctly.
7. Check that the armrests are correctly mounted, function properly, and are not broken.
8. Check that the back in plastic or netweave is correctly mounted and intact.
9. Check that the footrests are intact and can be adjusted.
10. Check the function of the bucket holder.

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EN:

This product should be recycled following national regulation. After disassembly of the product, sort the components based on what they are made of and recycle them individually. Please contact your local recycling place for guidance if necessary.

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